

SwiftMR

User Manual

Product Model

A20-CL

Version v2.0.1.X

AIRS
MEDICAL

SwiftMR
User Manual, English
A20-CL-UM-ID-EN Revision 1 (2022-09)
©2022 AIRS Medical Inc.
All rights reserved.

Contents

1. Introduction.....	3
1.1. Product Overview.....	3
1.2. Intended Use	3
1.3. Limitations.....	3
1.4. Functionalities	3
1.5. Minimum Requirements for the Software Operating Environment.....	4
1.6. Labeling.....	4
1.7. Warnings and Cautions.....	4
1.8. Cautions	5
1.9. Abbreviations.....	5
2. Functions	6
2.1. Login and Logout.....	6
2.2. Automated Logout	7
2.3. Main Page.....	7
2.4. Image Processing Status.....	9
2.5. Toast Message.....	10
2.6. Settings.....	12
2.6.1. General Settings.....	12
2.6.2. Appearance Settings.....	13
2.6.3. Profile Settings.....	14
2.6.4. Device Settings.....	15
2.6.5. Protocol Settings.....	16
2.6.6. Info Settings.....	18
3. User Account Management	18
4. MR Scans and Image Processing	18
4.1. MRI Scans	18
4.2. Image Processing	18
5. Cybersecurity	19
6. Trouble Shooting	19
7. Maintenance.....	20
8. Quality Assurance	20

Thank you for using AIRS Medical's Product.

AIRS Medical Inc. is a company devoted to developing products that exceed customer expectations as well as meet the relevant standards and legal requirements by aiming for world-class excellence in all tasks we perform.

This user manual provides instructions for using SwiftMR, including warnings and cautions to prevent hazardous situations. Please read this User Manual thoroughly before use.

Homepage

For more information about AIRS Medical and our products, please visit us at www.airsmmed.com.

General Information

- 『SwiftMR』 is a registered trademark of AIRS Medical Inc.
- The content of this User Manual is protected by copyright. If the content of this User Manual is modified or distributed without the written consent of AIRS Medical Inc., you will be liable for legal responsibilities.
- AIRS Medical Inc. may change or modify the specifications of the product and the contents of the manual without prior notice.

1. Introduction

1.1. Product Overview

SwiftMR provides automated image quality enhancement for MR images acquired from various environments. This device should be used only for professional purposes.

SwiftMR's automated processes are as follows:

- Uploading MR images in DICOM format after completions of MR scans
- Enhancing image quality using deep learning model
- Downloading enhanced MR images in DICOM format

Integration of PACS inside the institution with the manufacturer's Cloud server is necessary at the first installation of this product. Afterwards, SwiftMR performs image processing in the background automatically and provides logged in users the authorization to use functions and view image processing results in a worklist format through Client Application.

1.2. Intended Use

Software used in devices that store, enlarge, reduce, view, transmit, process, and output medical images. However, separately classified software is excluded here.

1.3. Limitations

SwiftMR is only available in certain countries.

This product meets the regulation of MoH of Indonesia for medical device software.


1.4. Functionalities

- Worklist view
 - Search by Scan date
 - Search by Patient name and/or Patient ID
 - Search by Modality
 - Search by Body part
 - Search by Status of image processing
- Image processing
 - Import acquired MR images from PACS or MR Device
 - Enhance image quality
 - Download the enhanced MR images to PACS

1.5. Minimum Requirements for the Software Operating Environment



Item	Launcher	Client Application
OS	Microsoft Window 10 64 bit	N/A
Hardware	CPU: Intel i3 RAM: 4GB and above Storage: 2GB available Memory: 500MB available	N/A
Web Browser	N/A	Google Chrome 92.0.4515 version or later Microsoft Edge 92 version or later
Network	Ethernet-based Intranet/internet connection	
Monitor	N/A	1600X900 resolution with 8-bit color

1.6. Labeling

Symbol	Definition
	'User Manual' reference symbol. This product is provided with the electronic instructions for use (eIFU). After signing up and being approved by the administrator at the AIRS Medical's eIFU homepage (https://en.airsmcd.com/eifu), you can view the user manuals via PDF Reader.

1.7. Warnings and Cautions

Warnings and cautions are for the correct and safe usage of the product. Please be advised of all the following contents for safety. AIRS Medical Inc. is not responsible for failures due to negligence of warnings and cautions.

Symbols	Definition
Note	Indicates useful information about features of the software.
Caution 	Indicates potentially hazardous situations for the patient or user that could result in lost time, reduced image quality and/or re-examination of the patient.
Warning 	Indicates potentially hazardous situations that could result in direct or indirect patient injury, mainly in the form of misinterpretation or misdiagnosis.

1.8. Cautions



- **Warning**

- SwiftMR is designed for trained radiology technologists. Those who are not trained should not use this product.
- Images that have already been quality-enhanced should not be processed again. This may cause damage to the images.



- **General Caution**

- Installation, maintenance, and repair of SwiftMR must only be done by engineers qualified and certified by AIRS Medical Inc.
- Operation and maintenance of SwiftMR strictly follows the user manual. Please preserve this user manual well.



- **Caution for Use**

- Check before use to see if the Quality Check, that the system conducts automatically at a specified time each day or conducted manually, has successfully been completed.
- Do not turn off the power of the PC or programs during use.
- After use, be sure to log out of the program.

1.9. Abbreviations

The following is the definitions of the abbreviations used in this User Manual.

DICOM	Digital Imaging and Communications in Medicine
MRI	Magnetic Resonance Imaging
PACS	Picture Archiving and Communication System

2. Functions

Client Application can be accessed through Google Chrome or Microsoft Edge browser in the hospital where SwiftMR is installed. You can access it through [SwiftMR server's IP address in the hospital]: 50000/swiftmr-client or the shortcut icon created during the installation process.

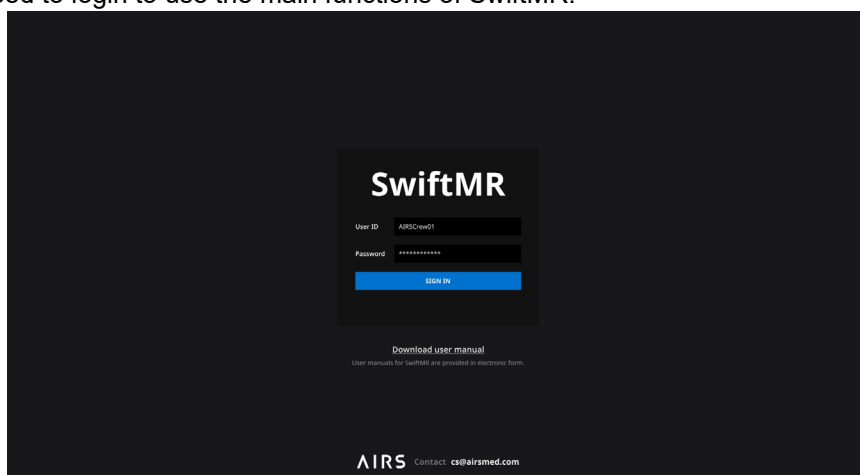



SwiftMR

The main functions of the Client Application are as follows.

2.1. Login and Logout

You need to login to use the main functions of SwiftMR.



- **Login:** Enter ID and password provided by AIRS Medical and then click **SIGN IN**. When login succeeds, the main page of SwiftMR will appear.
- **Logout:** You can log out from SwiftMR to prevent unauthorized access by clicking the Logout button ( Logout). When logout succeeds, you will be directed to the login page.
- **User Manual:** You can access the e-IFU download page of AIRS Medical by clicking on the “Download user manual” button ([Download user manual](#)).

Note

Note: User account is created by **AIRS Medical**. After an account is created, you will be able to use it after resetting the password upon first login.



Caution: If login fails because an invalid ID or password was entered into the input field, an error message will show up. If you forget your ID or password, contact Customer Support (cs@airsmmed.com).



Caution: SwiftMR does not allow simultaneous logins of the same account. If you try to login to an already logged-in account, the previously accessed session will be terminated.



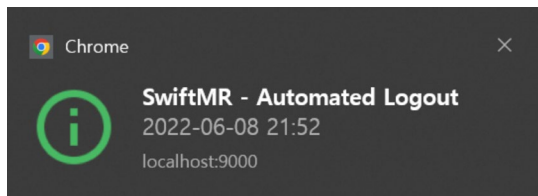
Caution: When login attempts fail 10 times, the user is restricted from login for the next 5 minutes.



Caution: Please contact Customer Support (cs@airsmmed.com) if you cannot access the eIFU download page.

2.2. Automated Logout

Users inactive for a certain period will be automatically logged out from SwiftMR to prevent unauthorized access. In the event of an automated logout, a Toast Message will appear and the screen will return back to the login page.



The Toast Message for automated logout disappears after a certain period, or it can be removed by clicking the **X** button at the top right corner.

2.3. Main Page

Main page of SwiftMR is constructed as follows:

- SwiftMR loads the list of acquired MR images from the connected PACS and display it in a worklist format.
- Users can easily search for an image of interest, and view detailed information, processing status, and processing results of the listed MR images from the worklist that is updated periodically.
- Users can check image processing results of each MR image in the Detailed Info section.

SwiftMR | ABC Gwanak | AIRSAdmin01 | MR 1 | Profile | Settings | S/W Info | Logout

Date Range: 2020-11-19 to 2020-11-26 | Patient ID: Enter patient ID | Patient Name: Enter patients name | Modality: MR | Body Part: Brain | Status: [Search]





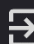
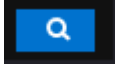
Study ID	Study Date	Patient ID	Patient Name	Sex	Age	Date of Birth	Requested Procedure	Study Description	Number of Series	Number of Instances	Accession Number	Status
1	2020-11-26 17:37:21	12341234	KIM HYEON JEONG	F	25	1997-04-24	MRA-BRAIN	AIRS BRAIN	6	192	201112341234	
2	2020-11-26 17:37:21	12341234	KIM HYEON JEONG	F	25	1997-04-24	MRA-BRAIN	AIRS BRAIN	6	192	201112341234	
3	2020-11-26 17:37:21	12341234	KIM HYEON JEONG	F	25	1997-04-24	MRA-BRAIN	AIRS BRAIN	6	192	201112341234	Completed
4	2020-11-26 17:37:21	12341234	KIM HYEON JEONG	F	25	1997-04-24	MRA-BRAIN	AIRS BRAIN	6	192	201112341234	ERROR
5	2020-11-26 17:37:21	12341234	KIM HYEON JEONG	F	25	1997-04-24	MRA-BRAIN	AIRS BRAIN	6	192	201112341234	Not Supported
6	2020-11-26 17:37:21	12341234	KIM HYEON JEONG	F	25	1997-04-24	MRA-BRAIN	AIRS BRAIN	6	192	201112341234	Paused
7	2020-11-26 17:37:21	12341234	KIM HYEON JEONG	F	25	1997-04-24	MRA-BRAIN	AIRS BRAIN	6	192	201112341234	Completed
8	2020-11-26 17:37:21	12341234	KIM HYEON JEONG	F	25	1997-04-24	MRA-BRAIN	AIRS BRAIN	6	192	201112341234	Completed
9	2020-11-26 17:37:21	12341234	KIM HYEON JEONG	F	25	1997-04-24	MRA-BRAIN	AIRS BRAIN	6	192	201112341234	Completed
10	2020-11-26 17:37:21	12341234	KIM HYEON JEONG	F	25	1997-04-24	MRA-BRAIN	AIRS BRAIN	6	192	201112341234	Completed

Series Number	Series Date	Series Description	Protocol Name	Number of Images	Status
1	2020-11-26 17:37:21	AX_T2_Flair_Swift	AX_T2_Flair_Swift	24	Completed
2	2020-11-26 17:37:21	AX_T2_Flair_Swift	AX_T2_Flair_Swift	24	Completed
3	2020-11-26 17:37:21	AX_T2_Flair_Swift	AX_T2_Flair_Swift	24	Completed
4	2020-11-26 17:37:21	AX_T2_Flair_Swift	AX_T2_Flair_Swift	24	Completed
5	2020-11-26 17:37:21	AX_T2_Flair_Swift	AX_T2_Flair_Swift	24	Completed
6	2020-11-26 17:37:21	AX_T2_Flair	AX_T2_Flair	24	Not Supported
7	2020-11-26 17:37:21	AX_T2_Flair_Swift	AX_T2_Flair_Swift	24	Completed
8	2020-11-26 17:37:21	AX_T2_Flair_Swift	AX_T2_Flair_Swift	24	Completed
9	2020-11-26 17:37:21	AX_T2_Flair_Swift	AX_T2_Flair_Swift	24	Completed

Detailed Info
Enhancement Result
Status: Process paused
Processing start time: 2020-12-31 15:00
Processing end time:

Notification
• Daily QC Success
2020-11-06 06:30
• Completed - Patient ID 1234123412341 (243/243)
2020-11-06 06:30
• Error - Patient ID 1234123412341
2020-11-06 06:30
• Process Paused
2020-11-06 06:30
• Process Resumed
2020-11-06 06:30
• Completed - Patient ID 1234123412341 (243/243)
2020-11-06 06:30
• Completed - Patient ID 1234123412341 (243/243)
2020-11-06 06:30
• Completed - Patient ID 1234123412341 (243/243)
2020-11-06 06:30

Detailed functions are as follows.

No	Function	Item	Description
1	Status Bar		Network connectivity status
		ABC Gwanak	Institution name
		AIRSAdmin01	Name of the accessed user
		MR 1	Name of the connected MR device(s)
2	User Menu	 Profile	User Profile
		 Settings	Software settings
		 S/W Info	Software information
		 Logout	Logout
3	Filter/Search	Data Range	Function to search the worklist by date range.
		Patient ID	Function to search the worklist by patient ID
		Patient Name	Function to search the worklist by patient name
		Modality	Function to search the worklist by modality
		Body Part	Function to search the worklist by body part
		Status	Function to search the worklist by image processing status
			Button to execute the search by the selected conditions
4	MR Study Table	Study Date	Study scan date (Year-Month-Day-Time)
		Patient ID	Patient ID
		Patient Name	Patient name
		Sex	Patient sex
		Age	Patient age
		Date of Birth	Patient date of birth (Year-Month-Date)
		MR Device Name	Name of the MR device that created the study (based on SwiftMR registration)
		Requested Procedure	Requested procedure
		Study Description	Name of study protocol
		Number of Series	Total number of series in a study
		Number of Instances	Total number of images in a study
		Accession number	Unique identification number of each image
		Status	Image processing status <ul style="list-style-type: none"> • In Progress • Completed • Not Supported • Error
5	MR Series Table	Series number	Sequence number of the series within the study selected from the study table
		Series Date	Scan date of each series
		Series Description	Detailed description of the scanned sequences for each series
		Protocol Name	Protocol name of the scanned sequences for each series

		Number of Images	Total number of images in a series
		Status	Image processing status of the series
6	Detailed Info	Enhancement Result	Enhancement result <ul style="list-style-type: none"> • Status: Completed, Not supported, Error • Processing start time • Processing end time
7	Notification	Notification	Notification window displaying image processing status, Daily QC results, etc.



Caution: The network connection status icon shows the connection between the institution's network and the cloud server of SwiftMR. If a problem occurred with the internet connection of the PC running the software, you will not be able to check this through the network connection status icon, and you will be automatically logged out.



Warning: When network connection status icon is marked red, it may cause problems regarding image processing. Resolve the problem as soon as possible by contacting the IT manager or the manufacturer.



Caution: Worklist displays only the acquired MR images from the selected MR devices. Therefore, when you cannot find the desired study in the worklist, check the selected MR devices.



Caution: SwiftMR supports worklist synchronization with PACS for 7 days at maximum. Studies that have passed this amount of period, will be excluded from the worklist and will not be able to be inquired.



Caution: All personal information saved in SwiftMR will be anonymized 24 hours after registration. If you want to inquire patients' information about the MR studies processed by SwiftMR, please use a PACS viewer.

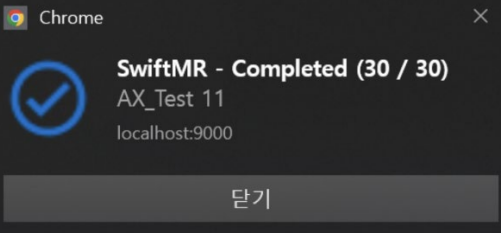
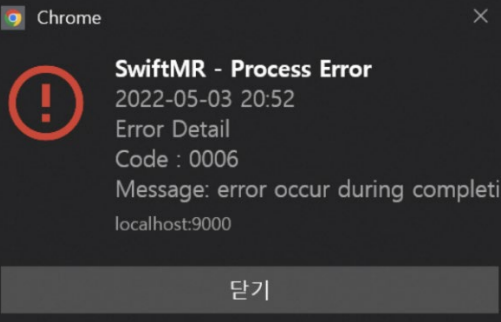
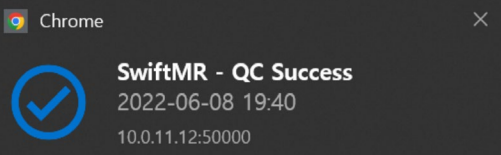
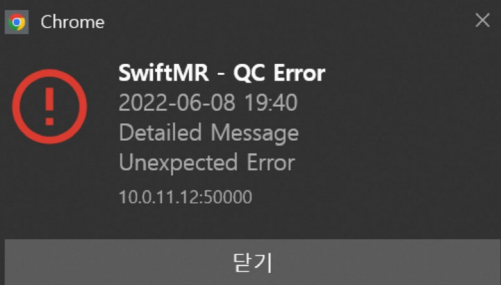
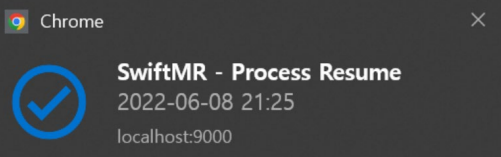
2.4. Image Processing Status

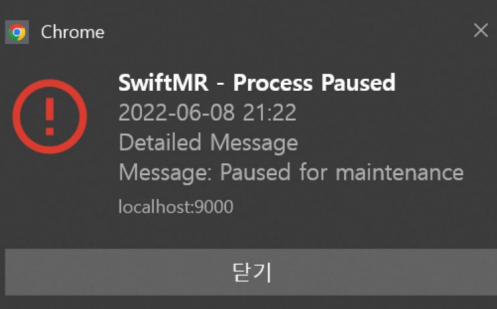
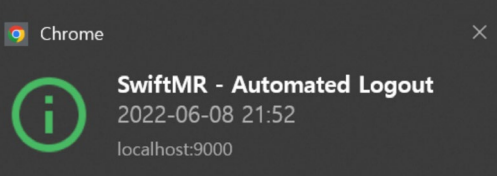
- Image processing status is classified as follows.

Icon	Description
	In progress Image processing progress is indicated with a progress bar
	Completed Image processing of all supported series in the study is successfully completed
	Error An error occurred during image processing.
	Paused Image processing is forcibly paused by the administrator while the image processing was in progress.
	Not supported There is no SwiftMR supported MR series in the study.

2.5. Toast Message

Users are notified of significant processing results with a toast message and through the Notification section. Toast message for each situation are as follows:

Toast Message	Description
	Image processing complete.
	Image processing failed.
	Daily QC succeeded.
	Daily QC failed.
	Image processing resumed.

 <p>Chrome</p> <p>SwiftMR - Process Paused 2022-06-08 21:22 Detailed Message Message: Paused for maintenance localhost:9000</p> <p>닫기</p>	Image processing paused.
 <p>Chrome</p> <p>SwiftMR - Automated Logout 2022-06-08 21:52 localhost:9000</p>	Automated logout.




Caution: Toast messages for the image processing results are only displayed for the connected MR devices. If images that are scanned through a desired device cannot be viewed, check the device setting in **Settings → Device**.



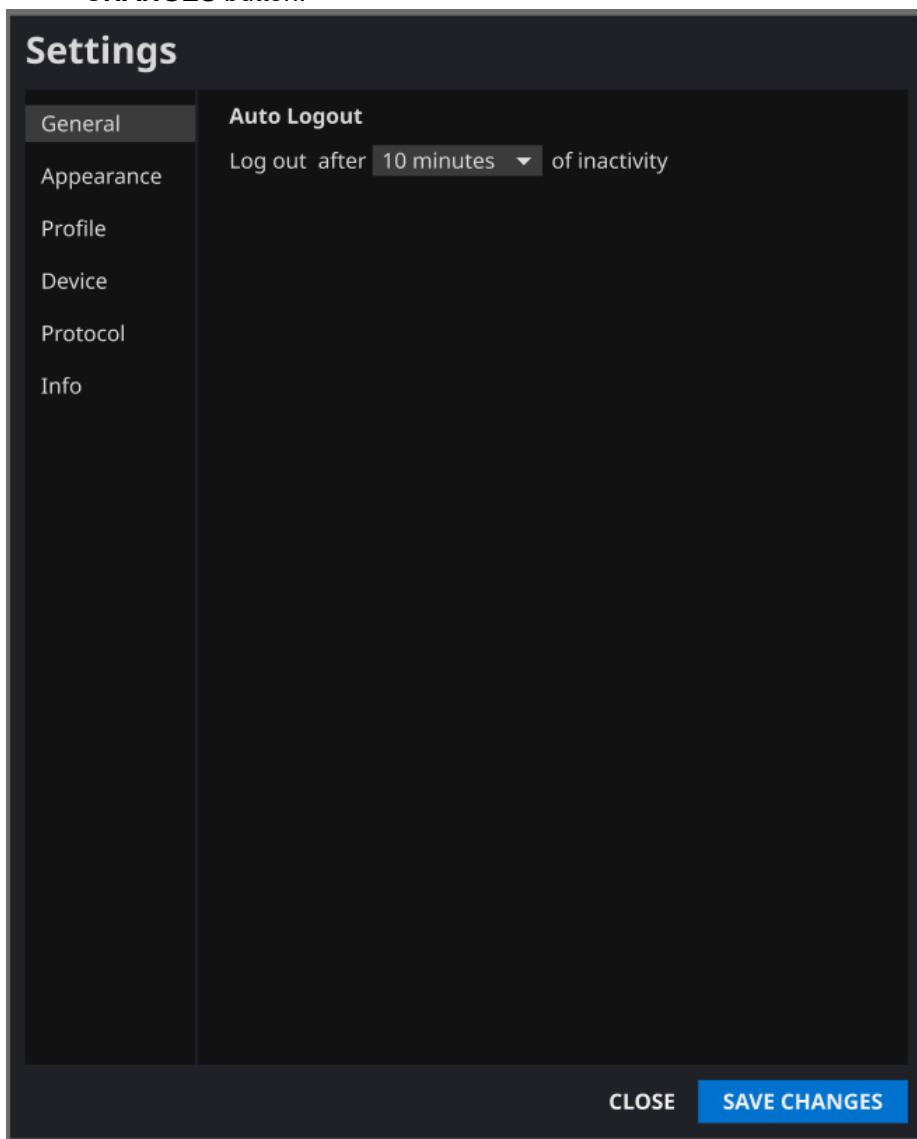
Caution: Toast message for image processing and image processing history in the Notification section is only shown for the connected MR devices. Therefore, if toast message for a desired study does not show up, please reset the device for which you want to be notified through the device settings in Section 2.6.4.

2.6. Settings

Users can open settings by clicking on the software preferences button ( Settings). Users can set various environment settings related to software operation in the software settings window. Software settings of the user consists of General, Appearance, Profile, Device, Protocol, and Info tabs.

2.6.1. General Settings

- **Auto logout time settings:** User can set the automated logout time from the General tab of Settings. Choose the time of the inactive period in which automated logout will occur afterwards from the drop-down menu and click on the **SAVE CHANGES** button.



2.6.2. Appearance Settings

- User can set the appearance of the main page from the Appearance tab of Settings.
 - **Font:**
 - Size – Choose from Small, Medium, and Large
 - Weight – Choose between Normal and Bold
 - **Colors:** Settings for the background color of the Toast Messages. Choose between Dark and Light.
 - **Table:** Select the columns to be shown in Worklist Table.
- After establishing the settings, click on the **SAVE CHANGES** button.

Setting

- General
- Appearance
- Profile
- Device
- Protocol
- Info

Font

Size Small Medium Large

Weight Normal Bold

SwiftMR Sample Text

Colors

Theme Dark Light

SwiftMR - QC Success
QC Message
OK

SwiftMR - QC Success
QC Message
OK

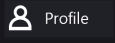
Table

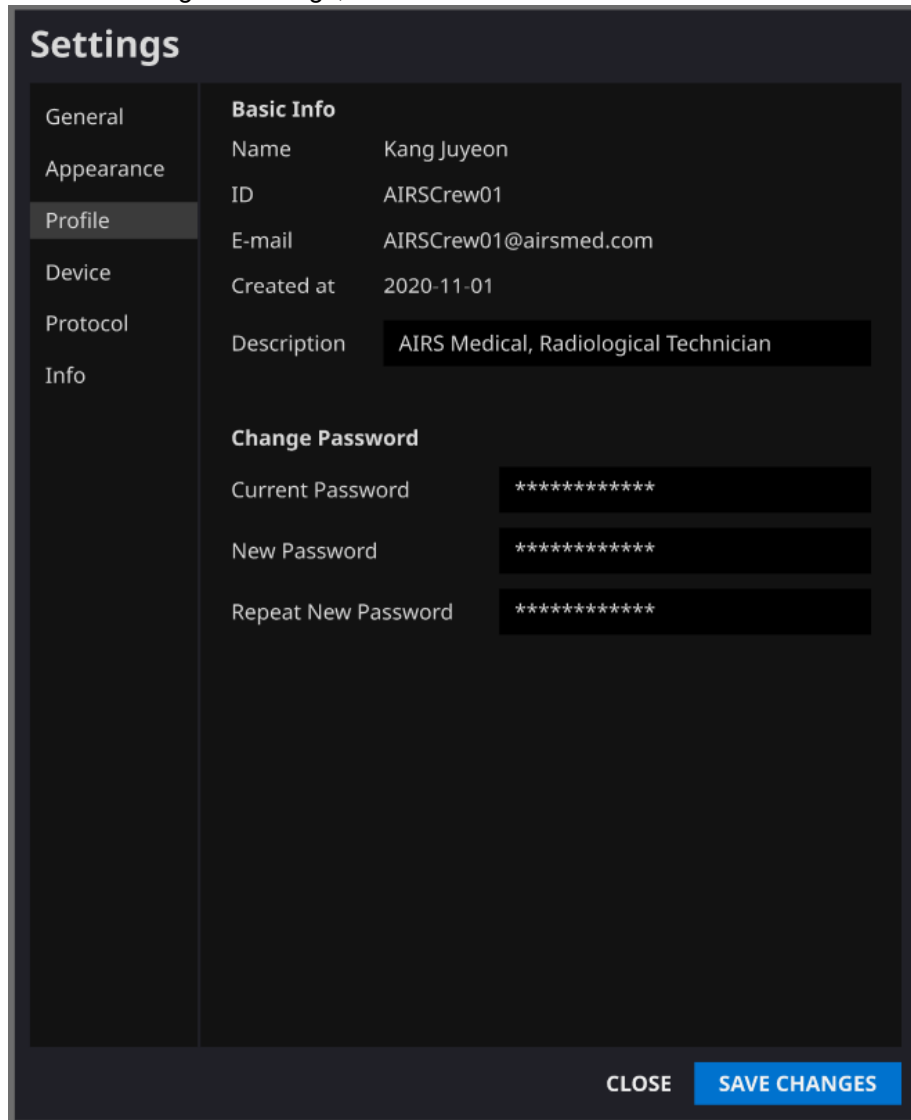
Column

- Study Date
- Patient ID
- Patient Name
- Age
- Sex
- Date of Birth
- MR Device Name
- Requested Procedure
- Study Description
- Number of Series
- Number of Instances
- Accession Number

CLOSE
SAVE CHANGES

2.6.3. Profile Settings

- Users can view or modify his/her own profile by clicking on the Profile button ( Profile) or from the Profile tab of Settings.
- Users can change his/her password by entering the previous and the new password.
- After establishing the settings, click on the **SAVE CHANGES** button.



Settings

General	Basic Info
Appearance	Name: Kang Juyeon
Profile	ID: AIRSCrew01
Device	E-mail: AIRSCrew01@airsmmed.com
Protocol	Created at: 2020-11-01
Info	Description: AIRS Medical, Radiological Technician

Change Password

Current Password: *****

New Password: *****

Repeat New Password: *****

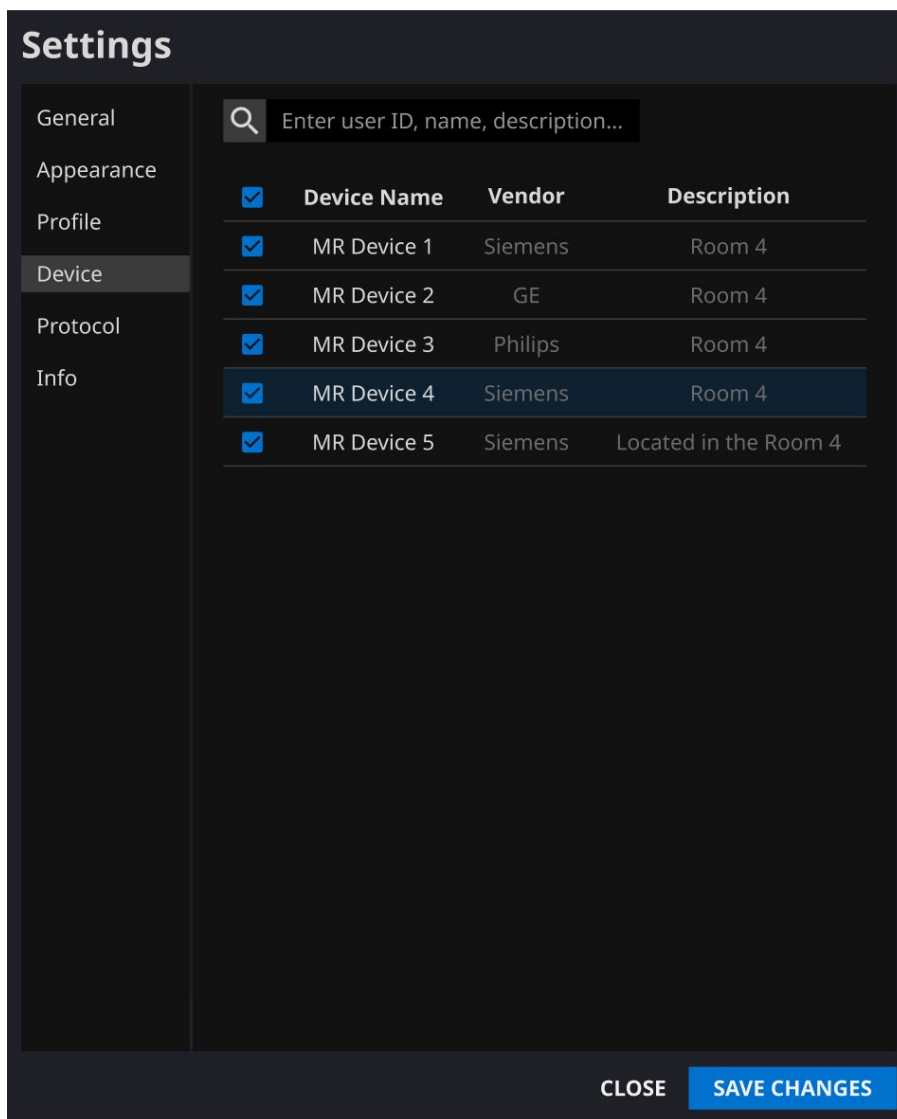
CLOSE **SAVE CHANGES**

Note: Only English is supported in the Description field.

Note: Password must satisfy the string rule of at least 8 characters including 1 letter, 1 number, and 1 special symbol.

2.6.4. Device Settings

- Users can select the MR devices to receive notifications for image processing results among the list of registered MR devices connected to the software's PACS from Device tab of Settings.
- All registered MR devices are selected by default.
- **Unselect:** Uncheck the device and click **SAVE CHANGES**.
- **Select:** Check the device and click **SAVE CHANGES**.
- After establishing the settings, click on the **SAVE CHANGES** button.
- Contact Customer Support (cs@airsmmed.com) to modify device settings.



Note: Each user can select multiple MR devices to receive notifications for.

2.6.5. Protocol Settings

- Users can view the SwiftMR image processing standard protocols, set for the applicable MR Device by selecting the desired device in the Protocol tab of Settings.
- SwiftMR takes the Protocol Name of an MR image to decide whether to process it. When processing is required, the enhancement will follow the Swift Model predefined in the Protocol setting.
- The Denoising Level and Sharpness Level to be applied during the image quality improvement process can be set through Customer Support.
- Please check the protocol for image processing before using SwiftMR.
- Contact Customer Support (cs@airsmmed.com) to modify protocol settings.

Settings

General

Appearance

Profile

Device

Protocol

Info

Select an MR Device
Select an MR device to view protocols.

MR Device 1	Siemens	10 Protocols	>
MR Device 1	Philips	1 Protocol	>
MR Device 1	GE	No Protocols	>

CLOSE

Settings

- General
- Appearance
- Profile
- Device
- Protocol
- Info

+ CREATE

Selected Device : MR Device 1 (Siemens)

Protocol Name	Swift Model	Sharpness
AX_T2_tse_Swift	TOF_HEAD:MIP4	5
AX_T2_tse_Swift	GENERAL	0
AX_T2_tse_Swift	GENERAL	0
SAG_T1_se_Swift	GENERAL	3

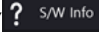
← SELECT DEVICE

CLOSE



Caution: If the input MR images were not obtained with the recommended MR imaging settings, there may be performance degradation in the resulting processed images.

2.6.6. Info Settings

Users can view user manual, information of the license in use, software information, and manufacturer information by clicking on the **S/W Info** () button or from the Info tab of Settings.

3. User Account Management

Passwords of user accounts can be changed in their profile settings (refer to Section 2.6.3). If you want to create a new user account or modify the basic information of an existing user account, please contact Customer Support (cs@airsmmed.com).

4. MR Scans and Image Processing

If you have completed the establishment of the settings to be suitable for the institution's circumstances by following the instructions of Section 2, you can now receive processed images by using SwiftMR.

4.1. MRI Scans

The imaging is performed using the MRI scan protocols provided by the manufacturer. Before scanning, select the appropriate MR Device in 'Settings → Protocol' from the Client Application, check whether the protocol name for the scan is registered and displayed on the UI, and then proceed with the MRI scan. When the scan is finished, the images are sent to the SwiftMR server through a specified path.

4.2. Image Processing

When images are received at the SwiftMR server, image processing is conducted using SwiftMR's deep learning model. The processed images are automatically saved in PACS in the form of DICOM files, and they can be saved as a new series in an existing study, or as an independent study that can be created and saved in PACS. The preference of this option will be defined during the discussion between the institution and the manufacturer prior to the contract.

When image processing is completed, this can be informed through a notification from the Client Application. SwiftMR does not provide a viewer program, so please use the existing DICOM image viewer used by the institution for viewing the images.

5. Cybersecurity

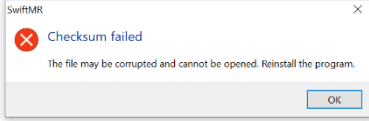

In order to ensure the cybersecurity of medical devices, SwiftMR meets the requirements of availability, confidentiality and integrity as follows.

- **Availability** implies that data should be readily available to an authorized user in the required format, whenever or wherever it is required.
- **Confidentiality** implies that data is not disclosed to an unauthorized person and is not used for unauthorized purposes. Even if data is disclosed by illegal methods such as an unauthorized inquiry or errors during data transmission/reception, SwiftMR makes it difficult to decrypt data through effective encryption and only allows authorized users to access the data. It also limits the range of data access according to the purpose and privileges of data users.
- **Integrity** implies that data is not altered or destroyed in any unauthorized manner. All information handled by SwiftMR should be accurate and complete, and the cybersecurity system of SwiftMR protects data from being distorted due to forgery or falsification. In addition, SwiftMR only allows authorized users to amend data handled in the system, and also manages logs and change history.

If you have any issues related to cybersecurity, please contact Customer Support (cs@airsmmed.com).

6. Trouble Shooting

If you have any problem that cannot be resolved through this user manual, please contact Customer Support (cs@airsmmed.com).

Category	Symptom	Cause & Measure
Integrity anomaly		Part of the installation files may be damaged or lost. Reinstall the program or contact the manufacturer.
Network Connection	Network connection status icon is 	This indicates that there is a communication problem between the hospital network and the SwiftMR cloud server. This may cause problems in image processing, thus contact your IT administrator or the manufacturer.
	Automatically logged out even when the set time for automatic logout has not elapsed.	There may be a problem with the Internet connection of the PC where the Client Application is installed. Check the Internet connection of your PC.
Worklist Synchronization	MR images that have been scanned are not imported into the SwiftMR worklist	This may be because the applicable MR Device is not connected to SwiftMR. Check the device settings at [Settings → Device].
		This may be because the Protocol Name used in the MR scan is different from the one registered in the protocol settings. Check the protocol settings at [Settings → Protocol].

7. Maintenance

Before using SwiftMR, please check to see if Daily QC has been completed successfully. When a problem is detected during use, the system will utilize the latest QC logs for conducting an abnormality check.

To fix bugs and to update image processing algorithms, it is recommended to upgrade the product at least once a year.

- In the case of the Cloud type model (A20-CL), all updates and upgrades are performed automatically on the server, so you can utilize the most up-to-date performance.

8. Quality Assurance

AIRS Medical Inc. warrants against defects in the design and production process during the warranty period of 1 year from the date of receipt of the product by the customer. In the event of a dispute, it will be proceeded in accordance with the Fair Trade Commission's settlements of consumer disputes.

AIRS Medical Inc. does not guarantee the following items:

- Damage caused by external factors such as accidents, misuse, fire, earthquake, etc.
- Modified products without the written consent of AIRS Medical Inc.
- Damage caused by service performed by an engineer or service provider not authorized by AIRS Medical Inc.

Before requesting a warranty service, please refer to the contents of this user manual first, and then contact us through Customer Support.

- Customer Support: cs@airsmmed.com

SwiftMR

AIRS Medical Inc.

8-9F, CS Tower, 1838, Nambusunhwan-ro, Gwanak-gu, Seoul, 08788 Korea

www.airsmmed.com

cs@airsmmed.com