

# SwiftMR

---

## User Manual

**Product Model**

A20-CL

Version v1.5.0.X(EU)



SwiftMR

User Manual, English

A20-CL-UM-EU-EN Revision 3 (2024-07)

©2024 AIRS Medical Inc.

All rights reserved.

## Contents

1. Introduction.....	4
1.1. Product Overview.....	4
1.2. Intended Use .....	5
1.3. Intended Patient Population .....	5
1.4. Intended Users.....	5
1.5. Contraindication .....	5
1.6. Intended Use Environment.....	5
1.7. Storage Condition .....	5
1.8. Medical Indication .....	5
1.9. Clinical Benefit.....	5
1.10. Supported MR and PACS devices.....	6
1.11. Processing/Protection of Sensitive Information .....	6
1.12. Limitations.....	6
1.13. Functionalities .....	7
1.14. Minimum Requirements for the Software Operating Environment.....	7
1.15. Labeling.....	7
1.16. Symbols Glossary.....	8
1.17. Precautions.....	9
1.18. Abbreviations.....	9
2. Functions .....	10
2.1. Login and Logout.....	10
2.2. Automated Logout .....	11
2.3. Main Page.....	12
2.4. Image Processing Availability .....	15
2.5. Image Processing Status.....	15
2.6. Toast Message.....	16
2.7. Settings.....	17
2.7.1. General Settings.....	17
2.7.2. Quality Check (QC) Settings.....	18
2.7.3. Appearance Settings.....	19

2.7.4. Profile Settings.....	20
2.7.5. Device Settings.....	21
2.7.6. Scan Item Settings.....	22
2.7.7. Info Settings.....	24
2.8. Image Processing.....	26
2.8.1. General Information.....	26
2.8.2. About Sharpness Increase Function.....	28
2.8.3. Caution – Image Artifacts.....	29
2.8.4. Note - TOF pulse sequence.....	30
3. User Account Management.....	31
4. Cybersecurity.....	31
5. Trouble Shooting.....	32
6. Maintenance.....	33
7. Incident Reporting.....	33
8. Quality Assurance.....	33

## Thank you for using AIRS Medical's Product.

AIRS Medical Inc. is a company devoted to developing products that exceed customer expectations as well as meet the relevant standards and legal requirements by aiming for world-class excellence in all tasks we perform.

This user manual provides instructions for using SwiftMR, including warnings and cautions to prevent hazardous situations. Please read this User Manual thoroughly before use.

## Homepage

For more information about AIRS Medical and our products, please visit us at [www.airsmmed.com](http://www.airsmmed.com).

## General Information

- 『SwiftMR』 is a registered trademark of AIRS Medical Inc.
- The content of this User Manual is protected by copyright. If the content of this User Manual is modified or distributed without the written consent of AIRS Medical Inc., you will be liable for legal responsibilities.
- User manuals for SwiftMR are provided in electronic forms (eIFU). Internet connection is essential to access the eIFU.
- AIRS Medical Inc. may change or modify the specifications of the product and the contents of the manual without prior notice.
- Please contact Customer Support ([support@airsmmed.com](mailto:support@airsmmed.com)) to request a paper copy of a user manual. Customers will receive a paper copy within seven business days upon receipt of the request.

## Certified Use

Usage of SwiftMR is subject to local law and conditions of the regulations,

- CE (European Union): SwiftMR is a CE-marked medical device that complies with the Regulation (EU) 2017/745 (MDR).

# 1. Introduction

## 1.1. Product Overview


SwiftMR provides automated image quality enhancement for MR images acquired from various environments. This device should be used only for professional purposes.

SwiftMR's automated processes are as follows:

- Uploading MR images in DICOM format after completions of MR scans
- Enhancing image quality using deep learning model
- Downloading enhanced MR images in DICOM format

Integration of PACS inside the institution with the manufacturer's Cloud server is necessary at the first installation of this product. Afterwards, SwiftMR performs image processing in the background automatically and provides logged in users the authorization to use functions and view image processing results in a worklist format through Client Application.

The figure below depicts the overall architecture of SwiftMR.

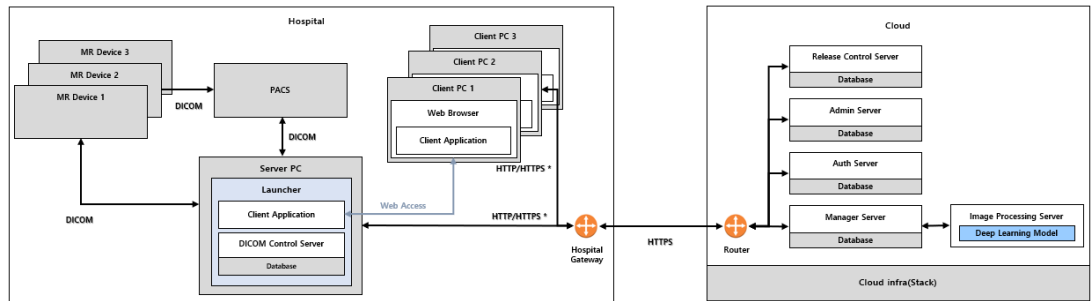

● HTTP/HTTPS\* : HTTP and HTTPS are used for in-hospital and external network communication, respectively

## 1.2. Intended Use

SwiftMR is a stand-alone software solution intended to be used for acceptance, enhancement and transfer of brain, spine, knee, ankle, shoulder and hip MR images in DICOM format. It can be used for noise reduction and increasing image sharpness for MR images.

SwiftMR is not intended for use on mobile devices.

## 1.3. Intended Patient Population

Adults over 21 years of age

## 1.4. Intended Users

**Health care professionals** (trained radiologic technologists)

- User should have experience with operating a software User Interface based on PC (including MRI and PACS)
- User should be able to understand terminology in the user manual and GUI in English

## 1.5. Contraindication

None.

## 1.6. Intended Use Environment

This product is to be used in the following care area:

- MRI room

## 1.7. Storage Condition

SwiftMR is stored on a cloud server.

## 1.8. Medical Indication

Patients with clinical conditions that need MR imaging of brain, spine, knee, ankle, shoulder or hip.

## 1.9. Clinical Benefit

SwiftMR provides diagnosis support.

Outcome parameters:

- Diagnostic quality
- SNR
- Image sharpness (FWHM)

## 1.10. Supported MR and PACS devices

The scope of MR devices supported by SwiftMR is as follows:

- Manufacturers: Siemens / GE / Philips
- Field Strength: 1.5T / 3.0T
- MR device models: All 1.5T / 3.0T models of the above 3 companies are supported.

The scope of PACS devices supported by SwiftMR is as follows:

- All DICOM-compliant PACS devices can communicate

## 1.11. Processing/Protection of Sensitive Information

### 1) Processing/Protection of User information (from user account)

User information such as name, ID, e-mail, and description are acquired during the SwiftMR user account creation process. These contents are stored in the AWS Cloud DB, where all information is encrypted.

### 2) Processing/Protection of patient sensitive Information

The DICOM images that were sent to SwiftMR for image processing are encrypted and stored in the DICOM Control Server's storage only for 6 hours from the time they are uploaded to the storage, while the images that failed are encrypted and stored for 7 days.

In order to display the study information that is being processed in the client application, the DICOM Control Server acquires the patient's name, ID, age, gender, and date of birth from the DICOM file that derived from the patient's scan. All information is encrypted and stored in the DICOM Control Server's DB.

In addition to this, when 24 hours have passed from the time of registration, the patient's name information is deleted from the DICOM Control Server's DB, while the Patient ID gets stored by applying a hash algorithm.

When DICOM Control Server and the Cloud exchange DICOM files containing sensitive information, the information is anonymized, and TLS security communication is applied. The cloud does not store any sensitive information.

## 1.12. Limitations

SwiftMR is only available in certain countries.

This product meets the EU MDR for medical device software.


### 1.13. Functionalities

- Worklist view
  - Search by Scan date
  - Search by Patient name and/or Patient ID
  - Search by Modality
  - Search by Body part
  - Search by Status of image processing
- Image processing
  - Import acquired MR images from PACS/MR Device
  - Enhance image quality
  - Download the enhanced MR images to PACS/MR Device

### 1.14. Minimum Requirements for the Software Operating Environment

Item	Launcher	Client Application
OS	Microsoft Window 10 64 bit	N/A
Hardware	CPU: Intel i3 and above RAM: 4GB and above Storage: 250GB available	N/A
Web Browser	N/A	Google Chrome 92.0.4515 version or later Microsoft Edge 92 version or later
Network	Ethernet-based Intranet/internet connection	
Monitor	N/A	1600X900 resolution with 8-bit color

### 1.15. Labeling











Symbol	Definition
	'User Manual' reference symbol. This product is provided with the electronic instructions for use (eIFU). After accessing AIRS Medical's eIFU homepage ( <a href="https://airsmc.com/eifu/">https://airsmc.com/eifu/</a> ), you can download the user manual within each countries' page and read it via PDF Reader.



## 1.16. Symbols Glossary

Symbols used in this user manual is as follows. Please familiarize yourself with the symbols in the table below.

Warnings, cautions and notes are for the correct and safe usage of the product. Please be advised of all the following for safety. AIRS Medical Inc. is not responsible for failures due to negligence of safety warnings and cautions.

Symbols	Meaning
<b>Note</b>	Indicates useful information about features of the software.
Caution 	Indicates potentially hazardous situations for the patient or user that could result in lost time, reduced image quality and/or re-examination of the patient.
Warning 	Indicates potentially hazardous situations that could result in direct or indirect patient injury, mainly in the form of misinterpretation or misdiagnosis.
	Manufacturer. The symbol is accompanied by the name and address of the manufacturer.
	e-IFU (electronic-Instructions for Use) The link that directs users to this user manual is alongside the symbol.
	Catalog number
	Lot number. Version number of the software.
	Indicates the product is a medical device.
	Unique Device Identifier
	Authorized representative in the European Community/European Union
	CE Mark Notified Body: DNV Product Assurance AS (2460)

### 1.17. Precautions



- **Caution for Use**

- Check before use to see if the Quality Check, that the system conducts automatically is completed successfully.
- Do not turn off the power of the PC or programs during use.
- After use, be sure to log out of the program.
- Images that have already been quality-enhanced should not be processed again. This may cause damage to the image.

- **Note Important Notes**

- SwiftMR is designed for trained radiology technologists.
- Installation, maintenance, repair and uninstallation of SwiftMR must only be done by engineers qualified and certified by AIRS Medical Inc.
- Operation and maintenance of SwiftMR strictly follow the user manual. Please keep this user manual.
- Contact Customer Support ([support@airsmc.com](mailto:support@airsmc.com)) to modify Daily QC time settings.

### 1.18. Abbreviations

The following is the definitions of the abbreviations used in this User Manual.

DICOM	Digital Imaging and Communications in Medicine
MRI	Magnetic Resonance Imaging
PACS	Picture Archiving and Communication System


## 2. Functions

Client Application can be accessed through Google Chrome or Microsoft Edge browser in the hospital where SwiftMR is installed. You can access it through [SwiftMR server's IP address in the hospital]: 50000/swiftmr-client.

The main functions of the Client Application are as follows.

### 2.1. Login and Logout

You need to login to use the main functions of SwiftMR.

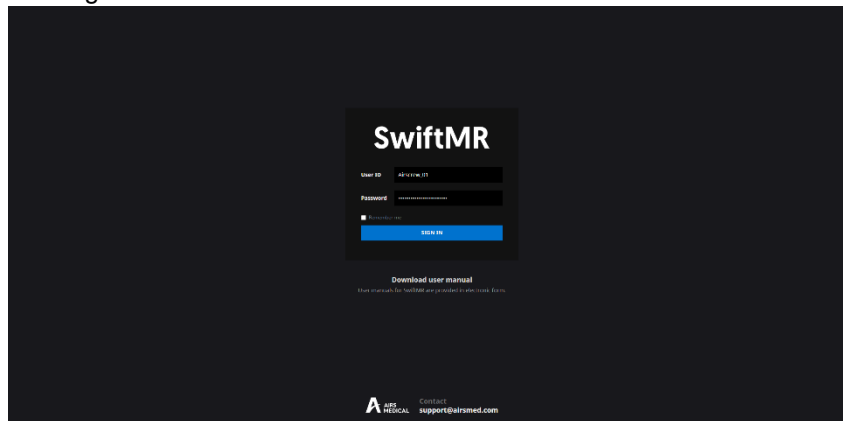


- **Login:** Enter ID and password provided by AIRS Medical and then click **SIGN IN**. When login succeeds, the main page of SwiftMR will appear.
- **Logout:** You can log out from SwiftMR to prevent unauthorized access by clicking the Logout button (Logout). When logout succeeds, you will be directed to the login page.
- **User Manual:** You can access the e-IFU download page of AIRS Medical by clicking on the “Download user manual” button (Download user manual).

<b>Note</b>	<b>Note:</b> User account is created by <b>AIRS Medical</b> . After an account is created, you will be able to use it after resetting the password upon first login.
<b>Note</b>	<b>Note:</b> If login fails because an invalid ID or password was entered into the input field, an error message will show up. If you forget your ID or password, contact Customer Support ( <a href="mailto:support@airsmc.com">support@airsmc.com</a> ).
<b>Note</b>	<b>Note:</b> SwiftMR does not allow simultaneous logins of the same account. If you try to login to an already logged-in account, the previously accessed session will be terminated.
<b>Note</b>	<b>Note:</b> When login attempts fail 10 times, the user is restricted from login for the next 5 minutes.
<b>Note</b>	<b>Note:</b> Please contact Customer Support ( <a href="mailto:support@airsmc.com">support@airsmc.com</a> ) if you cannot access the eIFU download page.

## 2.2. Automated Logout

Users inactive for a certain period will be automatically logged out from SwiftMR to prevent unauthorized access. In the event of an automated logout, a Toast Message will appear and the Client Application screen will return back to the login page, or it can be removed by clicking the OK button at the bottom.

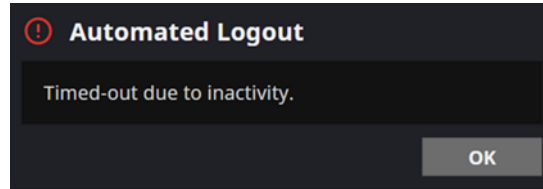


The Toast Message for automated logout disappears after a certain period.

### 2.3. Main Page


Main page of SwiftMR is constructed as follows:


- SwiftMR loads the list of acquired MR images from the MR Device and displays it in a worklist format.
- Users can easily search for an image of interest, and view detailed information, processing status, and processing results of the listed MR images from the worklist that is updated periodically.
- Users can check image processing results of each MR image in the Detailed Info section.
- Also, at the bottom of the main page, the license expiration, current usage, and maximum usage can be viewed.

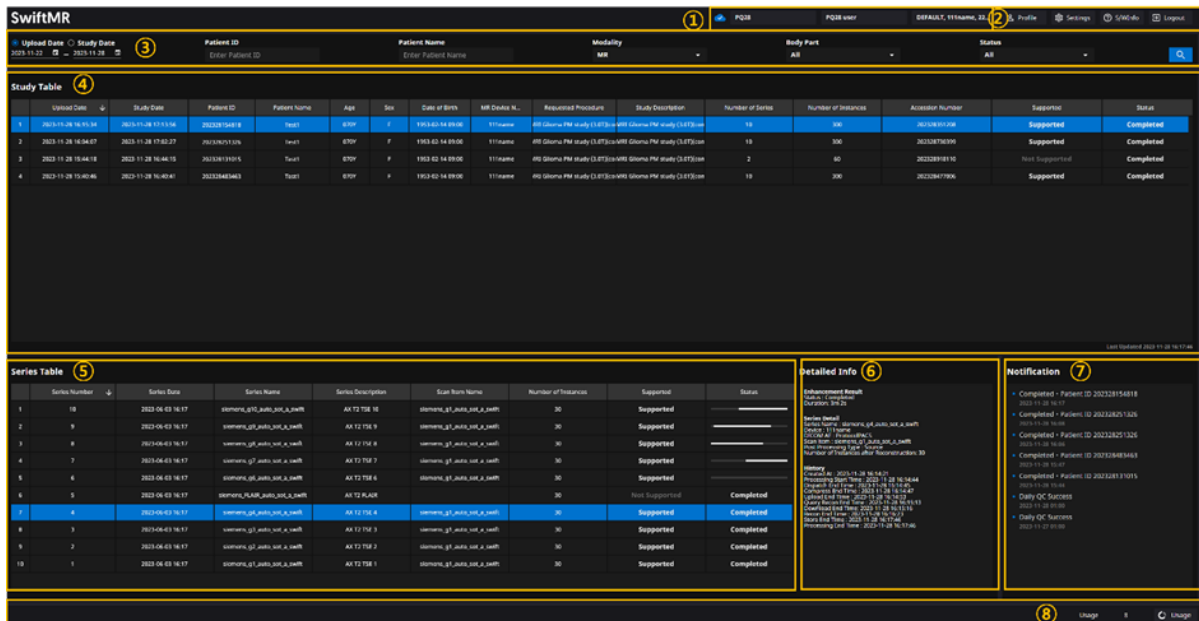

Detailed functions are as follows.

No	Function	Item	Description
1	Status Bar		Network connectivity status (Blue: Connected, Red: Disconnected, Grey: Connectivity Check Unavailable)
		ABC Gwanak	Institution name
		AIRSAdmin01	Name of the accessed user
		MR 1	Name of the connected MR device(s)
2	User Menu		User Profile
			Software settings
			Software information
			Logout

3	Filter/Search	Data Range	Function to search the worklist by date range.
		Patient ID	Function to search the worklist by patient ID
		Patient Name	Function to search the worklist by patient name
		Modality	Function to search the worklist by modality
		Body Part	Function to search the worklist by body part
		Status	Function to search the worklist by image processing status
			Button to execute the search by the selected conditions
4	MR Study Table	Upload Date	Upload scan date (Year-Month-Day)
		Study Date	Study scan date (Year-Month-Day)
		Patient ID	Patient ID
		Patient Name	Patient name
		Sex	Patient sex
		Age	Patient age
		Date of Birth	Patient date of birth (Year-Month-Date)
		MR Device Name	Name of the MR device that created the study (based on SwiftMR registration)
		Requested Procedure	Requested procedure
		Study Description	Name of study protocol
		Number of Series	Total number of series in a study
		Number of Instances	Total number of images in a study
		Accession number	Unique identification number of each image
		Supported	Image process support
Status	Image processing status <ul style="list-style-type: none"> <li>• In Progress</li> <li>• Completed</li> <li>• Error</li> </ul>		
5	MR Series Table	Series number	Sequence number of the series within the study selected from the study table
		Series Date	Scan date of each series
		Series Description	Detailed description of the scanned sequences for each series
		Protocol Name	Protocol name of the scanned sequences for each series
		Scan Item Name	Protocol name of the sequence registered in SwiftMR
		Number of Images	Total number of images in a series
		Supported	Image process support
		Status	Image processing status of the series
6	Detailed Info	Enhancement Result	Enhancement result <ul style="list-style-type: none"> <li>• Status: Completed, In Progress, Error</li> <li>• Duration: Processing time</li> </ul>
		Series Detail	<ul style="list-style-type: none"> <li>• Series Name: Processed sequence name</li> <li>• Device: MRI equipment name images are stored in</li> <li>• DICOM AE: PACS name images are stored in</li> <li>• Scan Item: Sequence protocol information registered in SwiftMR</li> </ul>

			<ul style="list-style-type: none"> <li>• Post Processing Type: Post processing type in the case of post processing images</li> <li>• Number Of Instances after Reconstruction: Number of slices in series that have been processed</li> </ul>
		History	<ul style="list-style-type: none"> <li>• Created At: Series transmission start time</li> <li>• Processing Start Time: Processing start time</li> <li>• Processing End Time: Processing end time</li> <li>• End time of other processing stages</li> </ul>
		Error Detail	<ul style="list-style-type: none"> <li>• Code: Error code</li> <li>• Message: Error Message</li> <li>• RetryCount: Number of retries attempted</li> </ul>
7	Notification	Notification	Notification window displaying image processing status, Daily QC results, etc.
8	Contract Information Status Bar	 Expiration date	Contract End Date
		Usage	Current Usage and Maximum Usage

**Note** **Note:** The network connection status icon shows the connection between the institution's network and the cloud server of SwiftMR. If a problem occurred with the internet connection of the PC running the software, you will not be able to check this through the network connection status icon, and you will be automatically logged out.





**Caution:** When network connection status icon is marked red or grey, it may cause problems regarding image processing. Resolve the problem as soon as possible by contacting IT Manager or the manufacturer.

**Note** **Note:** Worklist displays only the acquired MR images from the selected MR devices. Therefore, when you cannot find the desired study in the worklist, check the selected MR devices.

**Note** **Note:** Personal information stored in SwiftMR is anonymized 24 hours after registration. Use the PACS Viewer to inquire patient information about MR studies processed by the product.




## 2.4. Image Processing Availability

- Image processing availability is classified as follows.

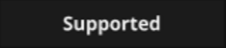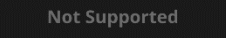Icon	Description
	<p><b>Image Processing - Supported</b></p> <ul style="list-style-type: none"> <li><b>Study Table:</b> If there is at least one SwiftMR-supported series among the series in the applicable study, the “Supported” icon will be displayed.</li> <li><b>Series Table:</b> If the series is supported by SwiftMR, then the “Supported” icon will be displayed.</li> </ul>
	<p><b>Image Processing – Not Supported</b></p> <ul style="list-style-type: none"> <li><b>Study Table:</b> If there is no SwiftMR-supported series among the series in the applicable study, the "Not Supported" icon is displayed.</li> <li><b>Series Table:</b> If the series is not supported by SwiftMR, then the “Not Supported” icon will be displayed.</li> </ul>

## 2.5. Image Processing Status





- Image processing status is classified as follows.

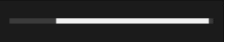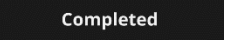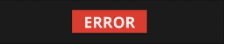Icon	Description
	<p><b>In progress</b></p> <p>Image processing progress is indicated with a progress bar</p>
	<p><b>Completed</b></p> <ul style="list-style-type: none"> <li><b>For Supported Studies:</b> Indicates that image processing has been completed and that the processed images are successfully stored to the selected DICOM AE (PACS, MR Device).</li> <li><b>For Not Supported Studies</b> Indicates that unsupported images are successfully stored to the selected DICOM AE (PACS, MR Device).</li> </ul>
	<p><b>Error</b></p> <p>An error occurred during image processing.</p>



## 2.6. Toast Message

Users are notified of significant processing results with a toast message and through the Notification section. Toast message for each situation are as follows:

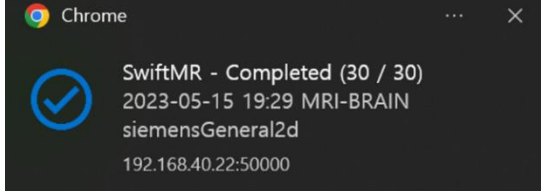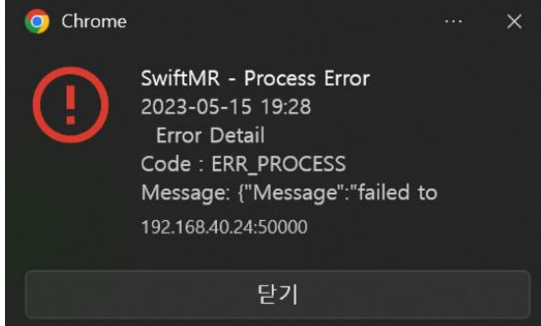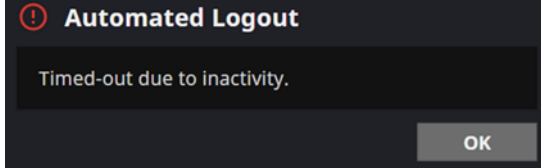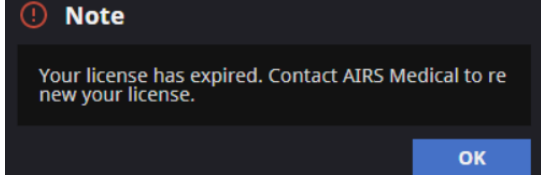Toast Message	Description
	Image processing complete.
	Image processing failed.
	Automated logout.
	End Date and Usage Expiration

---


**Note** **Note:** Toast messages for the image processing results are only displayed for the connected MR devices. If images that are scanned through a desired device cannot be viewed, check the device setting in Settings → Device.

---

**Note** **Note:** Toast message for image processing and image processing history in the Notification section is only shown for the connected MR devices. Therefore, if toast message for a desired study does not show up, please reset the device for which you want to be notified through the device settings in Section 2.7.5.


---

## 2.7. Settings

Users can open settings by clicking on the software preferences button (  Settings ). Users can set various environment settings related to software operation in the software settings window. Software settings of the user consists of General, QC, Appearance, Profile, Device, Scan Item, and Info tabs.


### 2.7.1. General Settings

- **Auto logout time settings:** User can set the automated logout time from the General tab of Settings. Choose the time of the inactive period in which automated logout will occur afterwards from the drop-down menu and click on the **SAVE CHANGES** button.

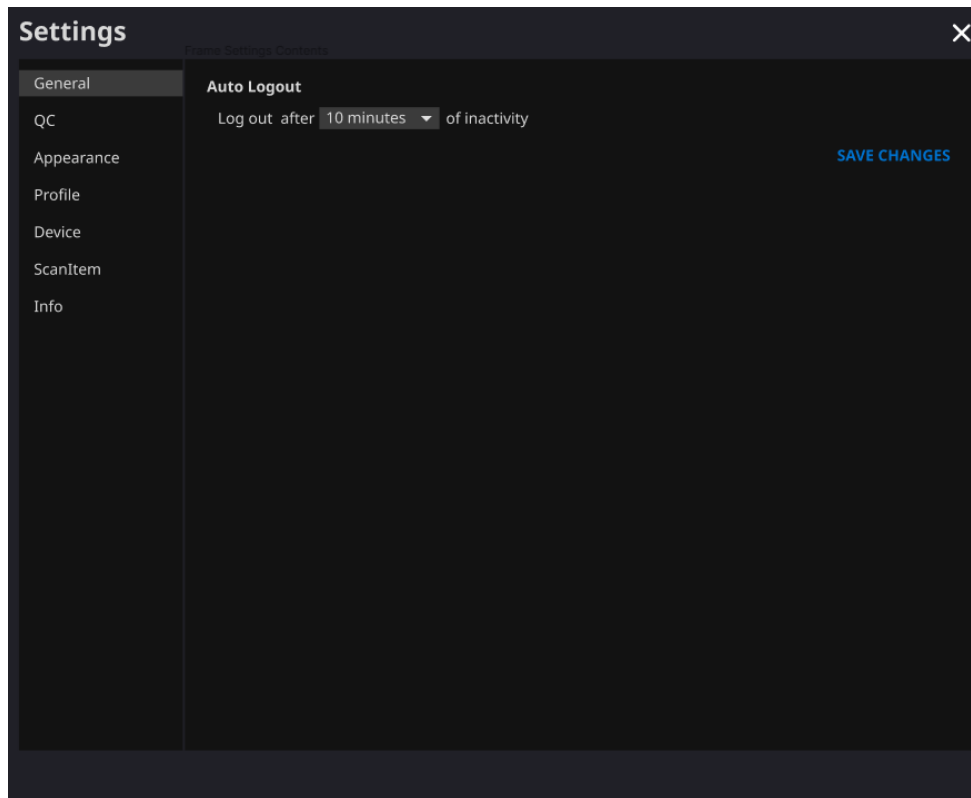


## 2.7.2. Quality Check (QC) Settings

- User can view the Quality Check settings and the results from the QC tab of Settings
  - **QC Status:** User can check the results and time of the most recently performed QC.
  - **Daily QC:** User can check how the QC time is set.
  - **QC Logs:** User can view the notifications regarding QC results.

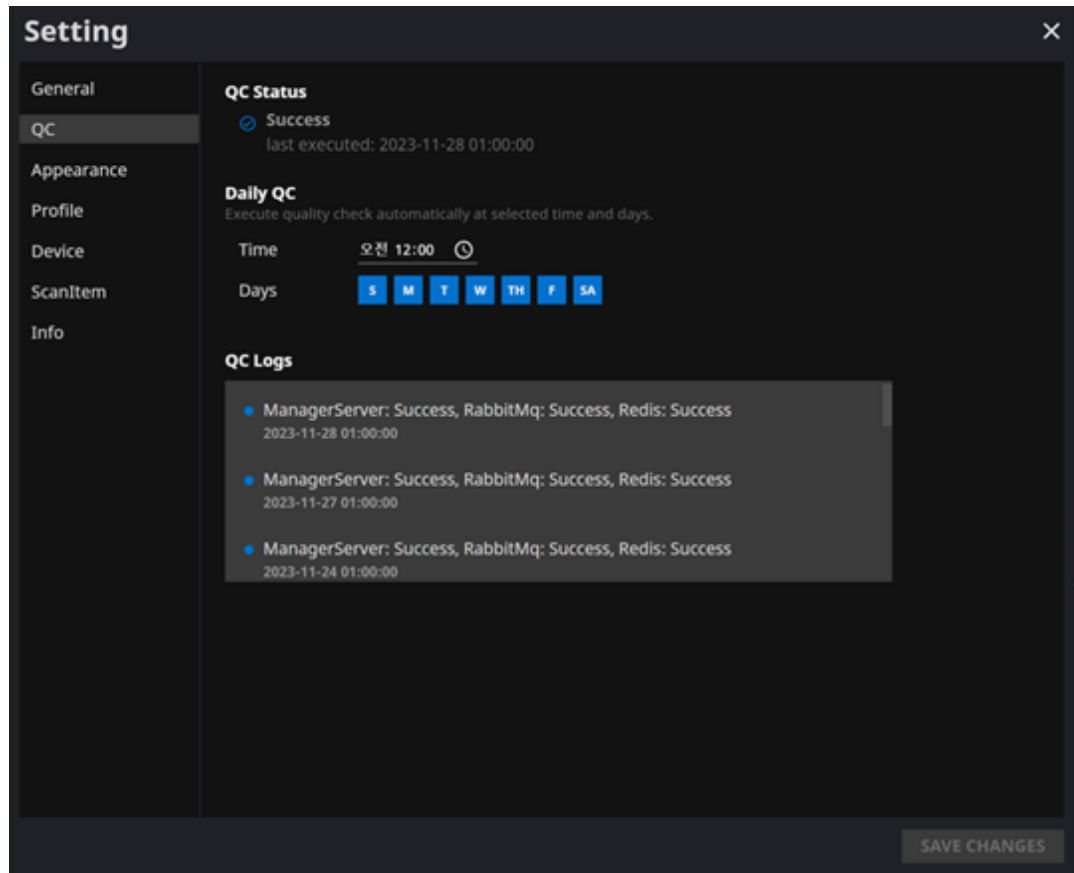



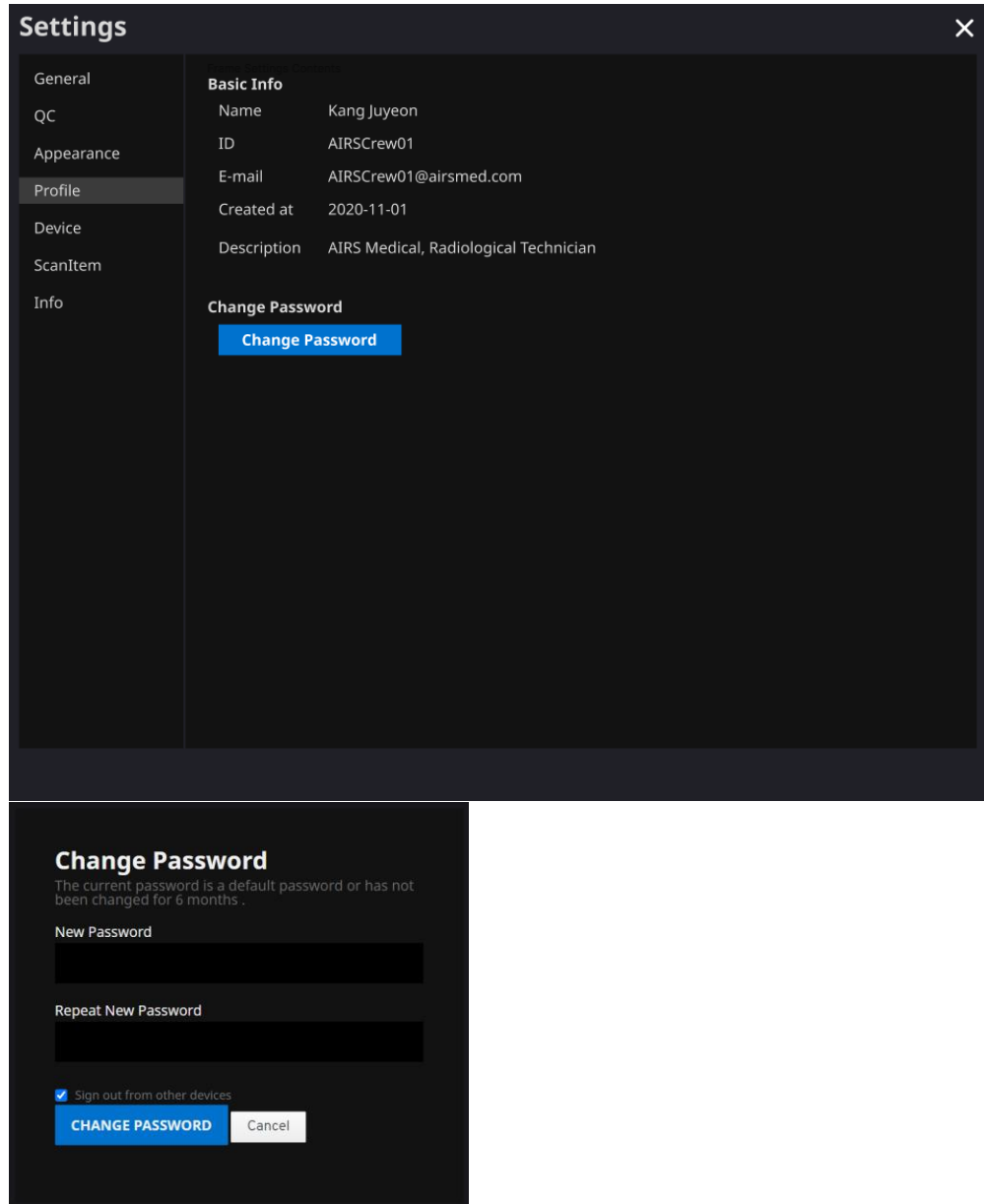
### 2.7.3. Appearance Settings

- User can set the appearance of the main page from the Appearance tab of Settings.
  - **Font:**
    - Size – Choose from Small, Medium, and Large
    - Weight – Choose between Normal and Bold
  - **Table:** Select the columns to be shown in Worklist Table.
- After establishing the settings, click on the **SAVE CHANGES** button.

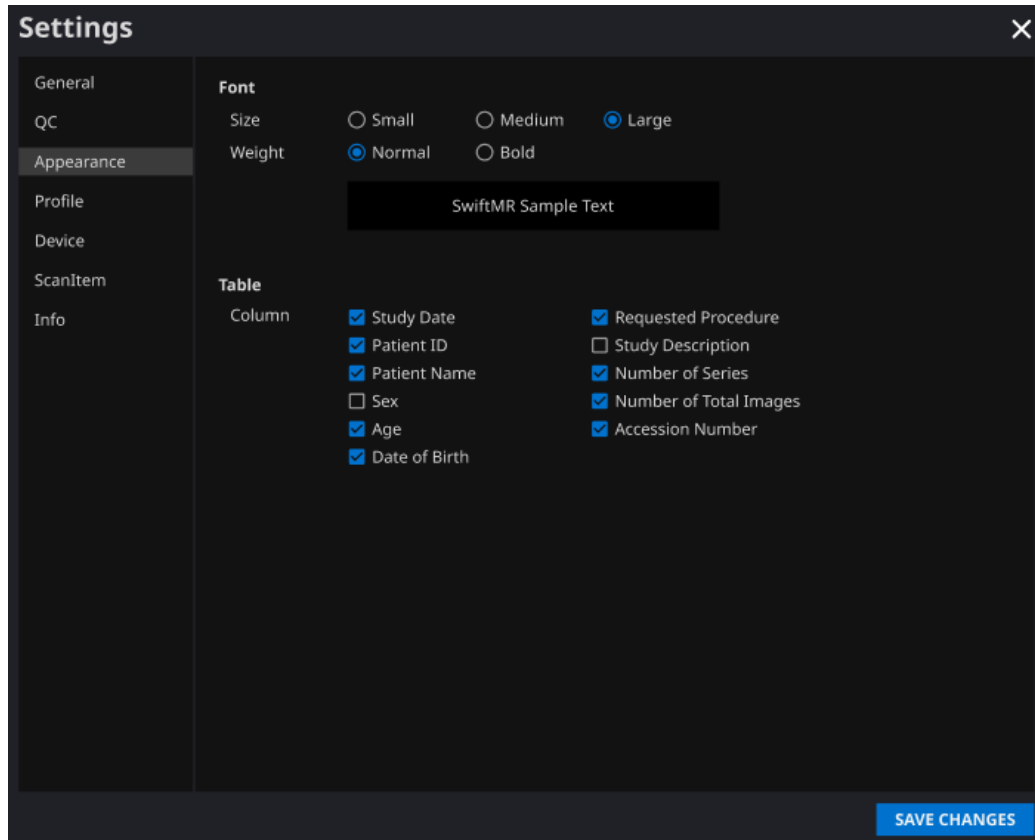

## 2.7.4. Profile Settings

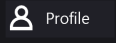- User can view or modify his/her own profile by clicking on the Profile button (  Profile ) or from the Profile tab of Settings.
- User can access the password change page through the password change button (  ) and change the user's password from the password change page.
- After establishing the settings, click on the **SAVE CHANGES** button.



The image shows two screenshots from a mobile application. The top screenshot is the 'Settings' screen with the 'Profile' tab selected. The 'Basic Info' section displays the following details:

Basic Info	
Name	Kang Juyeon
ID	AIRSCrew01
E-mail	AIRSCrew01@airsmmed.com
Created at	2020-11-01
Description	AIRS Medical, Radiological Technician

The 'Change Password' section contains a blue 'Change Password' button. The bottom screenshot is the 'Change Password' dialog box. It includes the following text and fields:

**Change Password**  
 The current password is a default password or has not been changed for 6 months .

New Password

Repeat New Password

Sign out from other devices


**CHANGE PASSWORD** Cancel

### Note

**Note:** Password must satisfy the password policy of having at least 12 characters including 1 upper-case letter, 1 lower-case letter, 1 number, and 1 special symbol.

### 2.7.5. Device Settings

- Users can select the MR devices to receive notifications for image processing results among the list of registered MR devices from Device tab of Settings.
- All registered MR devices are selected by default.
- **Unselect:** Uncheck the device and click **SAVE CHANGES**.
- **Select:** Check the device and click **SAVE CHANGES**.
- After establishing the settings, click on the **SAVE CHANGES** button.
- Contact Customer Support ([support@airsmmed.com](mailto:support@airsmmed.com)) to modify device settings.

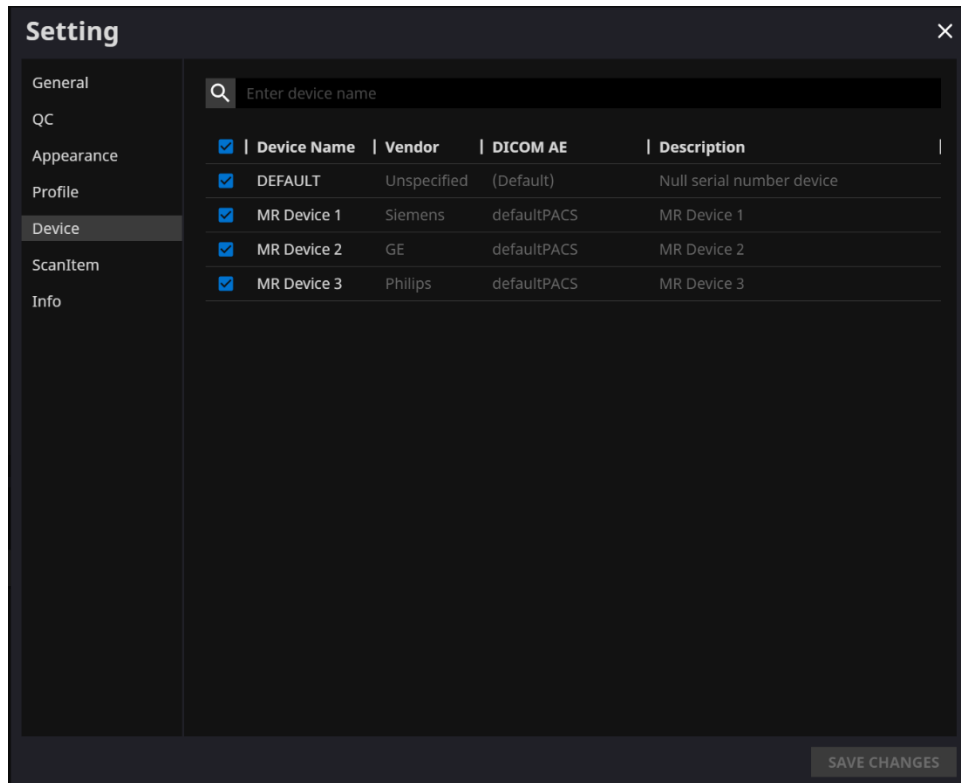



---

**Note**      **Note:** Each user can select multiple MR devices to receive notifications for.

---

### 2.7.6. Scan Item Settings

- Users can view the SwiftMR image processing standard protocols, set for the applicable MR Device by selecting the desired device in the Scan Item tab of Settings.
- SwiftMR takes the Scan Item Name of an MR image to decide whether to process it. When processing is required, the enhancement will follow the Swift Model predefined in the Protocol setting.
- The Sharpness Level to be applied during the image quality improvement process can be set through Customer Support.
- Please check the protocols for image processing before using SwiftMR.
- Contact Customer Support ([support@airsmmed.com](mailto:support@airsmmed.com)) to modify scan item settings.

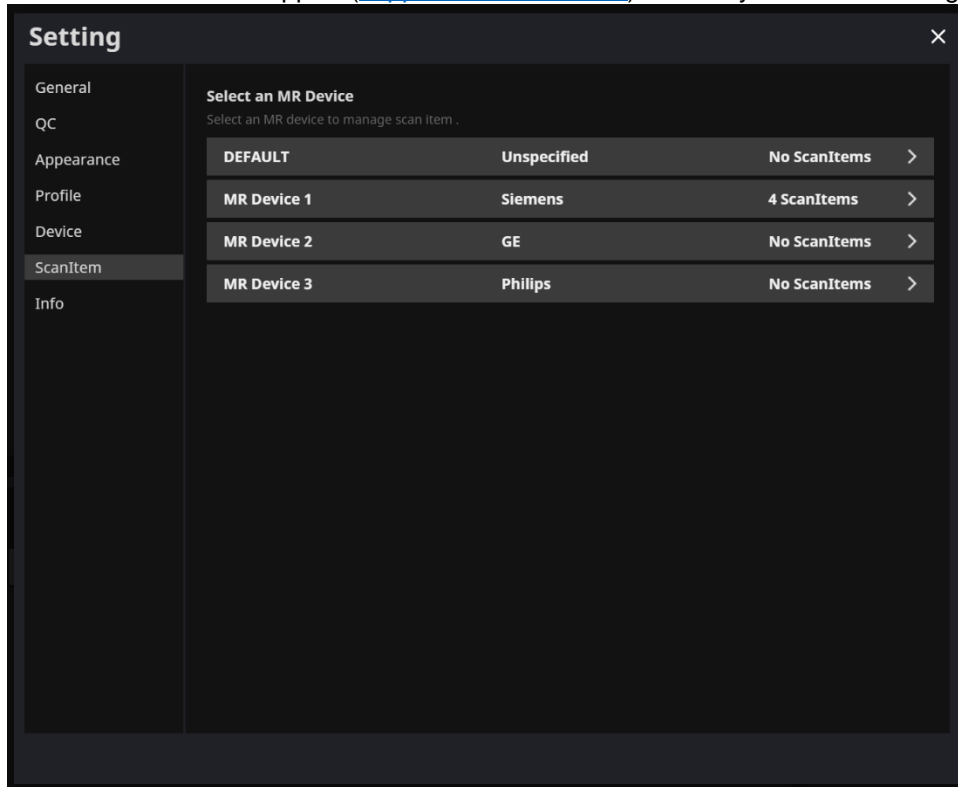

### Setting

- General
- QC
- Appearance
- Profile
- Device
- ScanItem
- Info

Enter scan item name, model...

Selected Device: MR Device 1 (Siemens) [← BACK TO LIST](#)


ScanItem Name	Swift Model	Sharpness
SAG_T2_se_Swift	Auto	0
SAG_T1_se_Swift	Auto	0
AX_T1_tse_Swift	Auto	0
AX_T2_tse_Swift	Auto	0




**Caution:** If the input MR images were not obtained with the recommended MR imaging settings, there may be performance degradation in the resulting processed images.



### 2.7.7. Info Settings

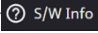Users can view user manual, information of the license in use, software information, and manufacturer information by clicking on the **S/W Info** (  ) button or from the Info tab of Settings.

In addition, the user can view the usage period information and maximum usage information.

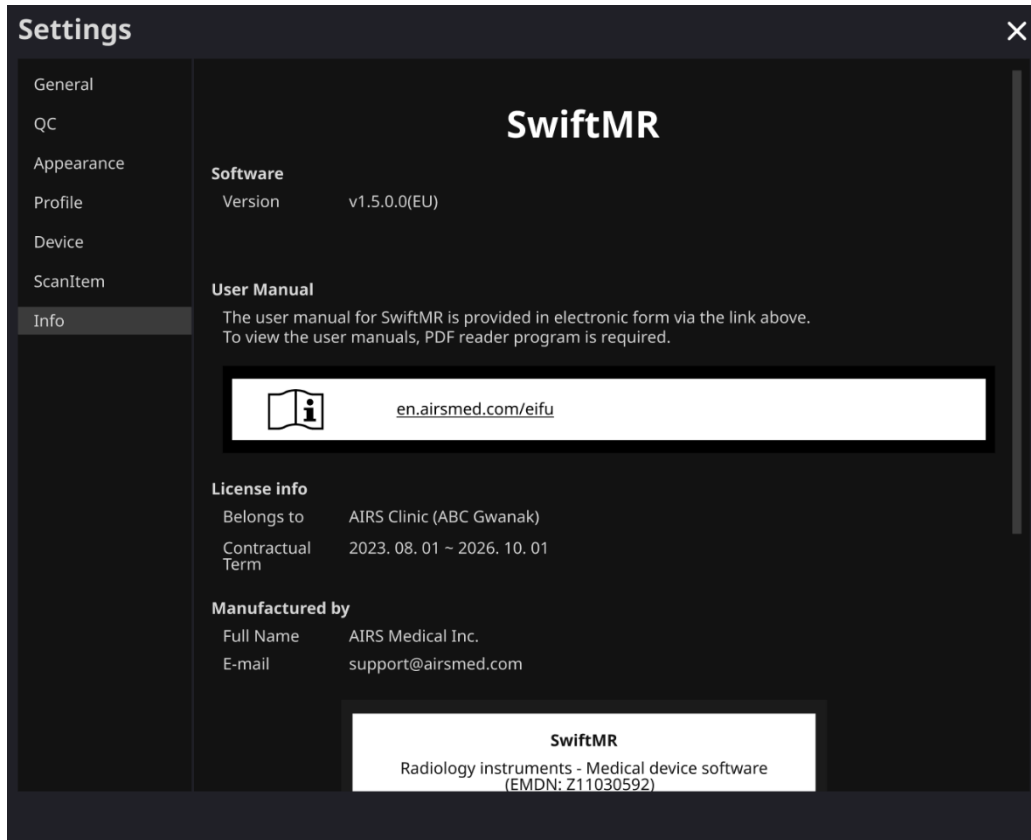

**Settings** ✕

- General
- QC
- Appearance
- Profile
- Device
- ScanItem
- Info**


## SwiftMR

**Software**

Version v1.5.0.0(EU)

**User Manual**

The user manual for SwiftMR is provided in electronic form via the link above. To view the user manuals, PDF reader program is required.

 [en.airsmmed.com/eifu](http://en.airsmmed.com/eifu)

**License info**

Belongs to AIRS Clinic (ABC Gwanak)

Contractual Term 2023. 08. 01 ~ 2026. 10. 01

**Manufactured by**

Full Name AIRS Medical Inc.

E-mail support@airsmmed.com

**SwiftMR**

Radiology instruments - Medical device software  
(EMDN: Z11030592)

✕


## Settings


- General
- QC
- Appearance
- Profile
- Device
- ScanItem
- Info

E-mail support@airsmmed.com


**SwiftMR**


Radiology instruments - Medical device software  
(EMDN: Z11030592)


**Model Name:** A20-CL    **Version Number:** v1.5.0.0(EU)    

 (01)08800124700099(10)v1.5.0.0(EU)


Storage condition: It is stored on a cloud server.

 **Name:** AIRS Medical Inc.  
**Address:** 13-14F, Keungil Tower, 223, Teheran-ro  
 Gangnam-gu, Seoul, 06142, Republic of Korea  
**Date of manufacture:** YYYY-MM-DD

 **Advena Ltd.** Tower Business Centre, 2nd Flr,  
 Tower Street, Swatar, BKR 4013 Malta

 **Advena Services Ltd**  
 Tower Business Centre, Tower Street  
 Swatar BKR 4013 Malta

Please contact Customer Support (support@airsmmed.com) to request a paper copy of a user manual. Customers will receive a paper copy within seven business days upon receipt of the request.


© 2024 AIRS Medical Inc.  
All rights reserved.

## 2.8. Image Processing

### 2.8.1. General Information

- The intended patient population is adults over 21 years of age.
- The imaging is performed using the MRI scan protocols provided by the manufacturer. Before scanning, select the appropriate MR device in 'Settings → Scan Item' from the Client Application, check whether the protocol name for the scan is registered and displayed on the UI, and then proceed with the MRI scan. When scan is finished, the images are sent to the SwiftMR server through a specified path. When subscription ends, use of the service will automatically terminate, and images processed through SwiftMR will not be able to be received.
- When images are received at the SwiftMR server, image processing is conducted using SwiftMR's deep learning model. The processed images are automatically saved in PACS in the form of DICOM files, and they can be saved as a new series in an existing study, or as an independent study that can be created and saved in PACS. The preference of this option will be defined during the discussion between the institution and the manufacturer prior the contract.
- When image processing is completed, this can be informed through a notification from the Client Application. SwiftMR does not provide a viewer program, so please use the existing DICOM image viewer used by the institution for viewing the images.
- SwiftMR applies the denoising function to the original image and additionally applies the sharpening function to it. The degree of noise reduction is single and the user cannot select it. Sharpening has 0-5 levels depending on the degree of sharpness increase.
- In level 0, the sharpening function is not applied to the image, only the denoising function is applied. In steps 1 to 5, the sharpening function is applied to the images that are already denoised by the deep learning model. The higher the sharpening level, the sharper the processed image. However, since the sharpening function sharpens the entire image, there is a risk of amplifying the residual artifacts included in the image as the sharpening level gets higher. In addition to this, the higher the sharpening level, the higher the image contrast compared to a normal MRI image, so it may look unnatural. Even if a low sharpening level is selected, a desired degree of image enhancement can be obtained depending on the user. When setting the protocol, the user should contact the Customer Support ([support@airsmed.com](mailto:support@airsmed.com)) to select the desired sharpening level.
- The sharpening level is not user selectable, but our service engineer selects it. The sharpening level can be set from level 0 to 5 irrespective of the imaging protocols. The initial value of sharpening level is set according to the user's request. If you want to change the sharpening level, please contact Customer Support ([support@airsmed.com](mailto:support@airsmed.com)).
- After image processing, the enhanced image is stored in PACS for default, and the original image is stored in PACS for optional. For enhanced Image, the phrase '**\_recon**' is added at the end of the protocol name and series description of the DICOM tag. Users can distinguish between original images and enhanced images by looking at the DICOM tag and depending on whether there is '**\_recon**'.
- The performance of SwiftMR both for noise reduction and sharpness increase was validated for the supported acquisition conditions. As a result of the test, it was demonstrated that SwiftMR increases the SNR of original image by 40% or greater than that on average. For sharpness, SwiftMR decreases FWHM of the tissue boundaries by 0.43% (level 1), 1.7% (level 2), 2.3% (level 3), 3.6% (level 4), 4.5% (level 5) or more for at least 90% of the dataset.

- Noise reduction performance was validated by comparing the original image and the image to which the denoising function was applied (ie, the image to which the sharpening function level 0 was applied).
- The scope of scanner models supported by SwiftMR is as follows.
  - Scanner manufacturers: Siemens / GE / Philips
  - Field Strength: 1.5T / 3.0T
  - Scanner models: All 1.5T / 3.0T models of the above 3 companies are supported.
- The range of bodyparts and pulse sequences supported by SwiftMR is as follows. For scan parameters such as tr, te, slice thickness, resolution, and acceleration factor, SwiftMR can be applied as long as it is within the range normally used when scanning SOC images for each bodypart and pulse sequence.

## Brain

Vendor	Field Strength	T1	T2	T2*	FLAIR	PD	MRA
Siemens	1.5T	v	v	v	v		v
	3.0T	v	v	v	v		v
GE	1.5T	v	v		v		
	3.0T	v	v	v	v		v
Philips	1.5T	v	v		v		
	3.0T	v	v	v	v		v

## Spine (consist of c-spine, t-spine, and l-spine)

Vendor	Field Strength	T1	T2	T2*	FLAIR	PD	MRA
Siemens	1.5T	v	v				
	3.0T	v	v				
GE	1.5T	v	v				
	3.0T	v	v				
Philips	1.5T	v	v				

## MSK (consist of shoulder, hip, knee, and ankle)

Vendor	Field Strength	T1	T2	T2*	FLAIR	PD	MRA
Siemens	1.5T	v	v			v	
	3.0T	v	v			v	
GE	1.5T	v	v			v	
	3.0T	v	v			v	
Philips	1.5T	v	v			v	
	3.0T	v	v			v	

### 2.8.2. About Sharpness Increase Function


- We used FWHM (Full Width Half Maximum) of structure boundaries as an index to validate the sharpness increase performance of the product. A decrease in FWHM means an increase in sharpness.
- The acceptance criteria for FWHM reduction rate for individual data was 0.43% (level 1), 1.7% (level 2), 2.3% (level 3), 3.6% (level 4), 4.5% (level 5). Although all test groups passed the criteria, the degree of sharpness increase may sometimes not be appreciable as the acceptance criteria values are not large.
- The average FWHM decrease rate and 95% CI for each test group are as follows:

Test Group	Avg FWHM decrease rate (%)
Siemens 1.5T (level 1)	9.95±5.93
Siemens 1.5T (level 2)	11.44±7.92
Siemens 1.5T (level 3)	19.70±8.87
Siemens 1.5T (level 4)	26.32±13.60
Siemens 1.5T (level 5)	46.04±32.41
Siemens 3.0T (level 1)	11.51±6.02
Siemens 3.0T (level 2)	13.67±8.43
Siemens 3.0T (level 3)	15.73±11.48
Siemens 3.0T (level 4)	25.50±13.86
Siemens 3.0T (level 5)	58.75±32.36
GE 1.5T (level 1)	9.54±6.25
GE 1.5T (level 2)	12.32±8.09
GE 1.5T (level 3)	20.80±10.40
GE 1.5T (level 4)	36.80±18.70
GE 1.5T (level 5)	48.15±31.51
GE 3.0T (level 1)	12.96±18.28
GE 3.0T (level 2)	16.09±20.14
GE 3.0T (level 3)	25.26±19.44
GE 3.0T (level 4)	26.93±21.28
GE 3.0T (level 5)	56.36±28.06
Philips 1.5T (level 1)	9.38±5.59
Philips 1.5T (level 2)	15.97±6.69
Philips 1.5T (level 3)	18.20±8.40
Philips 1.5T (level 4)	22.44±12.49
Philips 1.5T (level 5)	43.63±24.16
Philips 3.0T (level 1)	12.90±16.94
Philips 3.0T (level 2)	14.64±20.41
Philips 3.0T (level 3)	18.95±18.64
Philips 3.0T (level 4)	26.56±21.14
Philips 3.0T (level 5)	35.62±27.91

### 2.8.3. Caution – Image Artifacts



If image artifacts or processing errors are found in the SwiftMR processed image, user has the option to use the original image.

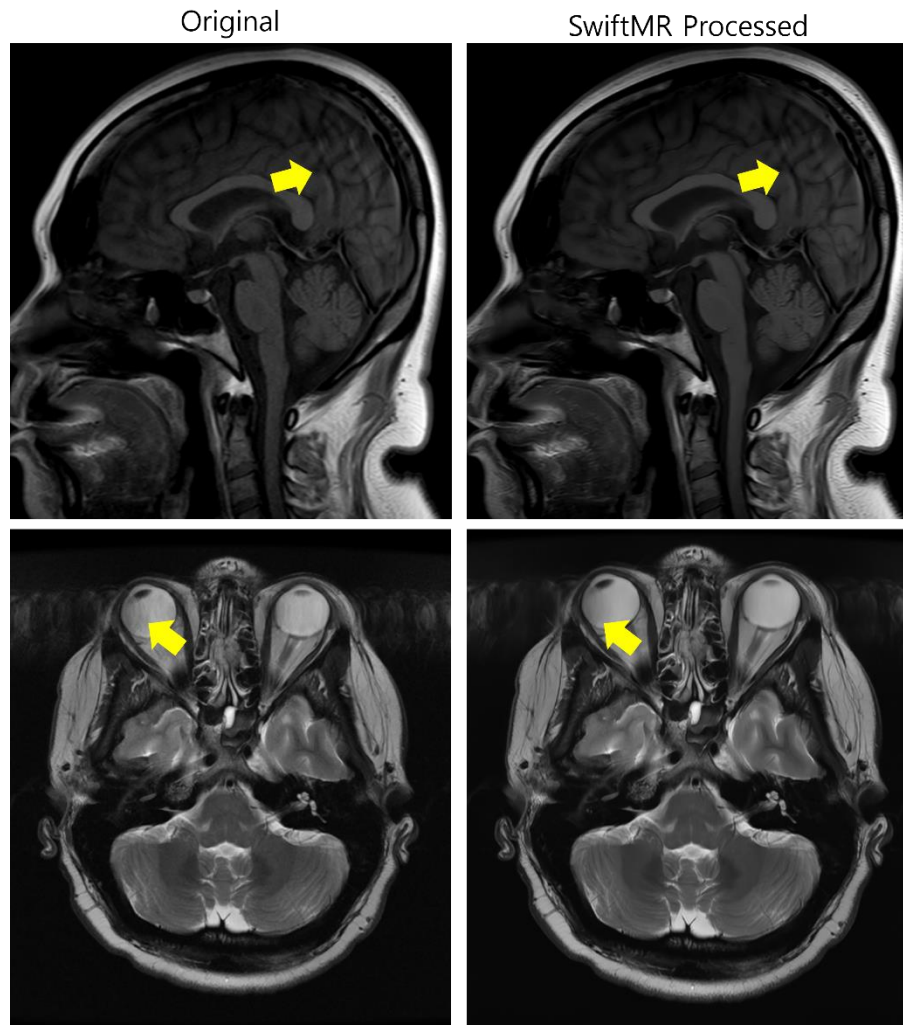

When image artifacts including motion artifacts, aliasing, noise, distortion, or ghosting exist in the original image, SwiftMR handles them in the same way as normal images. Due to the denoising / sharpening function of the SwiftMR process, there is a possibility that the artifacts of the original image may be reduced or, conversely, become more distinct.

Accordingly, if an image artifact is identified in the SwiftMR processed image, the user can use the original image. Original image and processed image can be distinguished through DICOM tag, see 2.8.1 for details.

#### 2.8.4. Note - TOF pulse sequence

**Note** For TOF pulse sequence slice images, it may seem that the vessel detail is not completely preserved after SwiftMR processing, but this is because the resolution is improved so that the cross section of the vessel spread out in the original image is seen narrower and more clearly in the processed image (Fig. (a), (b)). According to our validation results, vessel detail does not disappear due to SwiftMR processing, and users can check this by comparing MIP images before and after processing (Fig. (c), (d)).

However, user has the option to use the original image at any time, see 2.8.1 for details.

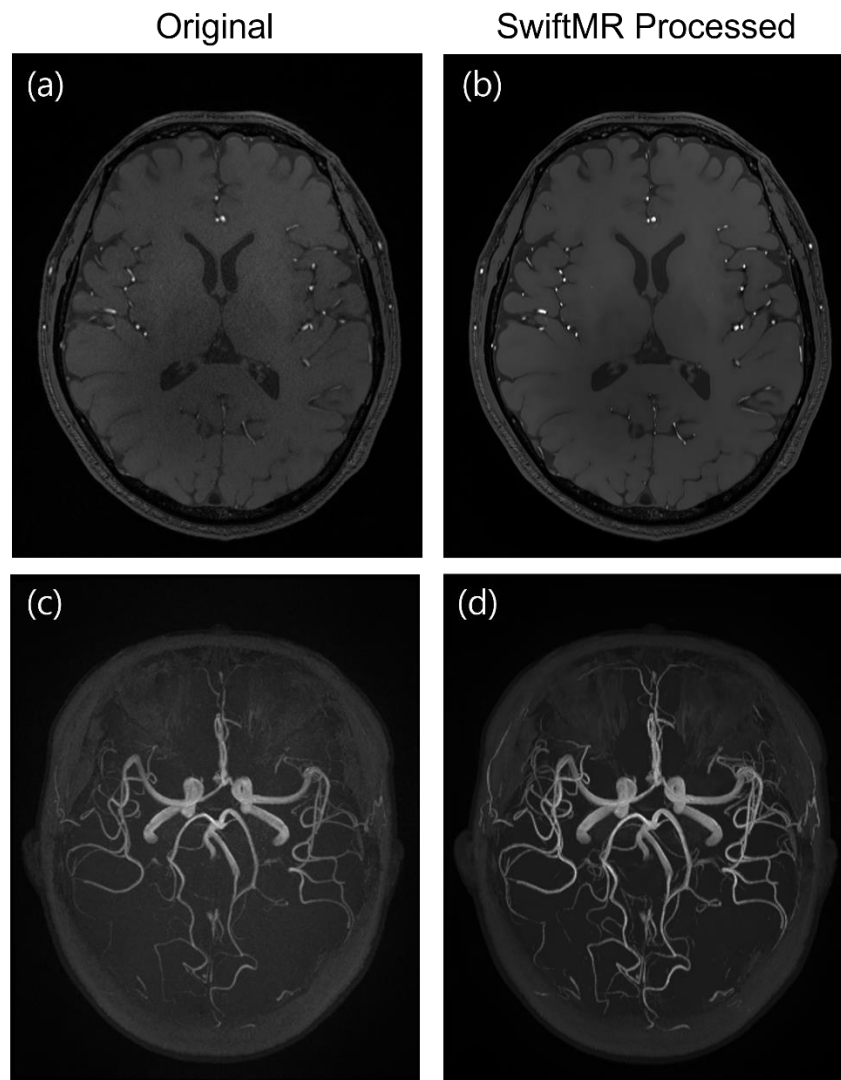


Figure. (a), (b): Slice image before and after SwiftMR processing. (c), (d): MIP image before and after SwiftMR processing

### 3. User Account Management

Passwords of user accounts can be changed in their profile settings (refer to Section 2.7.4). If you want to create a new user account or modify the basic information of an existing user account, please contact Customer Support ([support@airsmmed.com](mailto:support@airsmmed.com)).

### 4. Cybersecurity

In order to ensure the cybersecurity of medical devices, SwiftMR meets the requirements of availability, confidentiality and integrity as follows.

- **Availability** implies that data should be readily available to an authorized user in the required format, whenever or wherever it is required.
- **Confidentiality** implies that data is not disclosed to an unauthorized person and is not used for unauthorized purposes. Even if data is disclosed by illegal methods such as an unauthorized inquiry or errors during data transmission/reception, SwiftMR makes it difficult to decrypt data through effective encryption and only allows authorized users to access the data. It also limits the range of data access according to the purpose and privileges of data users.
- **Integrity** implies that data is not altered or destroyed in any unauthorized manner. All information handled by SwiftMR should be accurate and complete, and the cybersecurity system of SwiftMR protects data from being distorted due to forgery or falsification. In addition, SwiftMR only allows authorized users to amend data handled in the system, and also manages logs and change history.

Before installing and running SwiftMR, cybersecurity procedures must be performed according to the following guideline. The following guideline will help you protect this software from cybersecurity threats such as virus or breaches.

- Prior to installing and running SwiftMR, run a reliable anti-virus program to prevent data corruption from viruses.
- Keep your anti-virus software up-to-date.
- Confirm that the latest security updates are applied to your OS.
- Activate your PC's firewall. Windows 10 or above is equipped with a built-in firewall.
- SwiftMR has the highest possible security settings selected by default.
- When there is a new update, Users can choose whether to perform the update or not.
- When a forced update is required due to security issues or critical bugs, software update is mandatory. If not updated in a timely manner, the software may not function properly.
- When the software's cybersecurity has been compromised, AIRS Medical can disconnect the applicable network (VPN, TLS Proxy, etc.) to protect critical functionalities and data.
- The main settings required to operate SwiftMR are stored in the DICOM Control Server. Therefore, even if a problem occurs in the client application, and reinstallation is performed, the main settings are maintained for intact product operation. The database files of the DICOM Control Server are also backed up, and thus in case of an error emergence, the backed-up database files can be restored to make the product operational.
- Even if a temporary error occurs while using the Client Application, image processing will still operate normally, due to the functioning server programs. DICOM Control Server turns off when the PC shuts down, but it automatically is executed when the PC is turned on again. The images



that had not been processed during the time the DICOM Control Server was turned off, will be processed once they are resent to the server. In case of the server programs, uninterrupted operation is guaranteed via automatic restart or executing multiple instances during server shut down.


- Checksum validation is performed each time the SwiftMR application is executed, or when an update is conducted. Through this, the integrity check is performed, and if it is determined that there is a problem in the results, the program will automatically be terminated. In this case, please delete and reinstall the application or contact Customer Support ([support@airsmmed.com](mailto:support@airsmmed.com)).
- AIRS Medical is not responsible for accidents caused by not complying with the above instructions.

If you have any concerns or issues related to cybersecurity, use the phone number and email address listed at the last page to contact Customer Support.

If you have any issues related to cybersecurity, please contact Customer Support ([support@airsmmed.com](mailto:support@airsmmed.com)).

## 5. Trouble Shooting

If you have any problem that cannot be resolved through this user manual, please contact Customer Support ([support@airsmmed.com](mailto:support@airsmmed.com)).

Category	Symptom	Cause & Measure
Integrity anomaly	“Integrity test failed” message shows up and SwiftMR service is not executed.	Part of the installation files may be damaged or lost. Reinstall the program or contact the manufacturer.
Network Connection	Network connection status icon is 	This indicates that there is a communication problem between the hospital network and the SwiftMR cloud server. This may cause problems in image processing, thus contact IT Manager or the manufacturer.
	Automatically logged out even when the set time for automatic logout has not elapsed.	There may be a problem with the Internet connection of the PC where the Client Application is installed. Check the Internet connection of your PC.
Worklist Synchronization	MR images that have been scanned are not imported into the SwiftMR worklist	This may be because the applicable MR Device is not connected to SwiftMR. Check the device settings at [Settings → Device].

---

**Note**      **Note:** All activity and system logs of SwiftMR are stored and archived.

---

## 6. Maintenance

Before using SwiftMR, please check to see if Daily QC has been completed successfully. When a problem is detected during use, the system will utilize the latest QC logs for conducting an abnormality check.

To fix bugs and to update image processing algorithms, it is recommended to upgrade the product at least once a year.

- In the case of the Cloud type model (A20-CL), all updates and upgrades are performed automatically on the server, so you can utilize the most up-to-date performance.

## 7. Incident Reporting

In case of a serious incident, please contact us through Customer Support ([support@airsmmed.com](mailto:support@airsmmed.com)) and/or regulatory body that is applicable to the regulations where you have purchased and used SwiftMR.

## 8. Quality Assurance

AIRS Medical Inc. warrants against defects in the design and production process during the warranty period of 1 year from the date of receipt of the product by the customer. In the event of a dispute, it will be proceeded in accordance with the Fair Trade Commission's settlements of consumer disputes.

AIRS Medical Inc. does not guarantee the following items:

- Damage caused by external factors such as accidents, misuse, fire, earthquake, etc.
- Modified products without the written consent of AIRS Medical Inc.
- Damage caused by service performed by an engineer or service provider not authorized by AIRS Medical Inc.

Before requesting a warranty service, please refer to the contents of this user manual first, and then contact us through Customer Support.

- Customer Support: [support@airsmmed.com](mailto:support@airsmmed.com)

**SwiftMR** CE<sub>2460</sub>



**Advena Ltd.** Tower Business Centre, 2nd Flr.,  
Tower Street, Swatar, BKR 4013 Malta



**Advena Services Ltd**  
Tower Business Centre, Tower Street  
Swatar BKR 4013 Malta

**AIRS Medical Inc.**

13-14F, Keungil Tower, 223, Teheran-ro, Gangnam-gu, Seoul, 06142, Republic of Korea

Tel. (+82) 70 7777 3187

Fax. (+82) 02 6280 3185

[www.airsmmed.com](http://www.airsmmed.com)

[support@airsmmed.com](mailto:support@airsmmed.com)